

HARVEY®

Hydroclave MC8
Hydroclave MC10
Steam Sterilizer

PARTS CATALOG

Table of Contents

MC8 Exploded Views3-11

MC10 Exploded Views.....12-20


Accessory Parts21

 Non-Recirculating Option.....22


 Recommended Spare Parts List23

MC8


DETAIL OF TOP

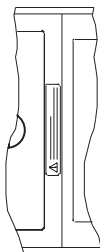


FRONT VIEW

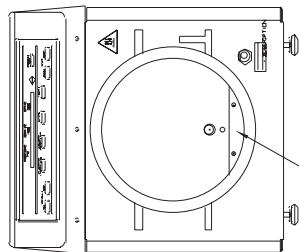

265848
(INSTALL DMTD PINS)

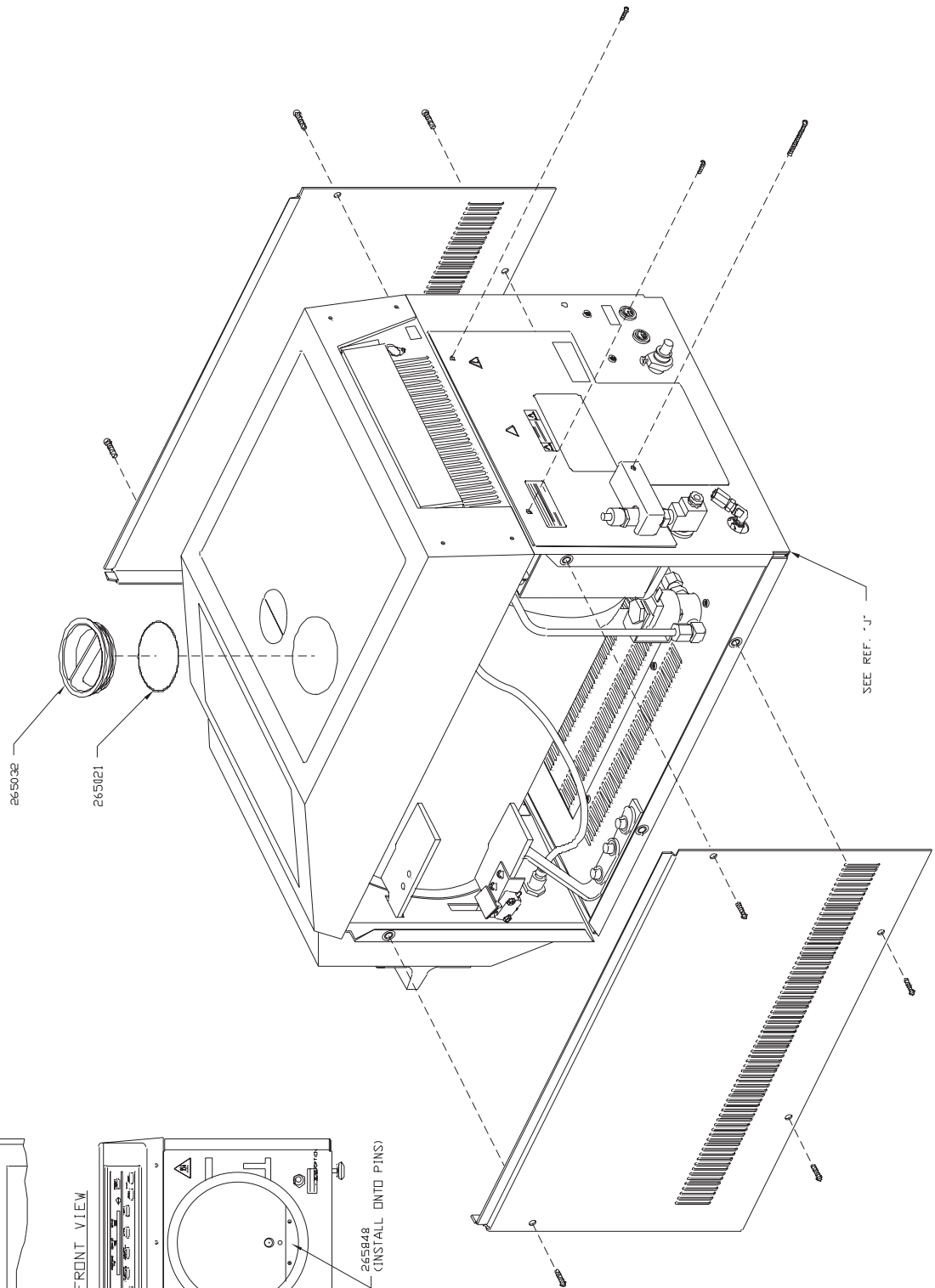

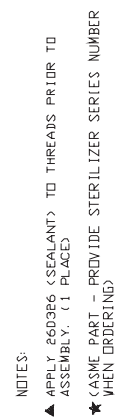

[illegible]



SOLENOID ASSEMBLY SHOULD PROTRUDE THROUGH HOLE ONCE ASSEMBLED

265757

250052401

PMX174

(2) 250052441

250052421

250010611

250052421


(4) 264667

265757


REF. "G"

NOTES:

▲ APPLY 260326 (SEALANT) TO THREADS PRIOR TO ASSEMBLY,
(2 PLACES).

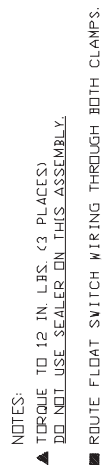


REF. "F"

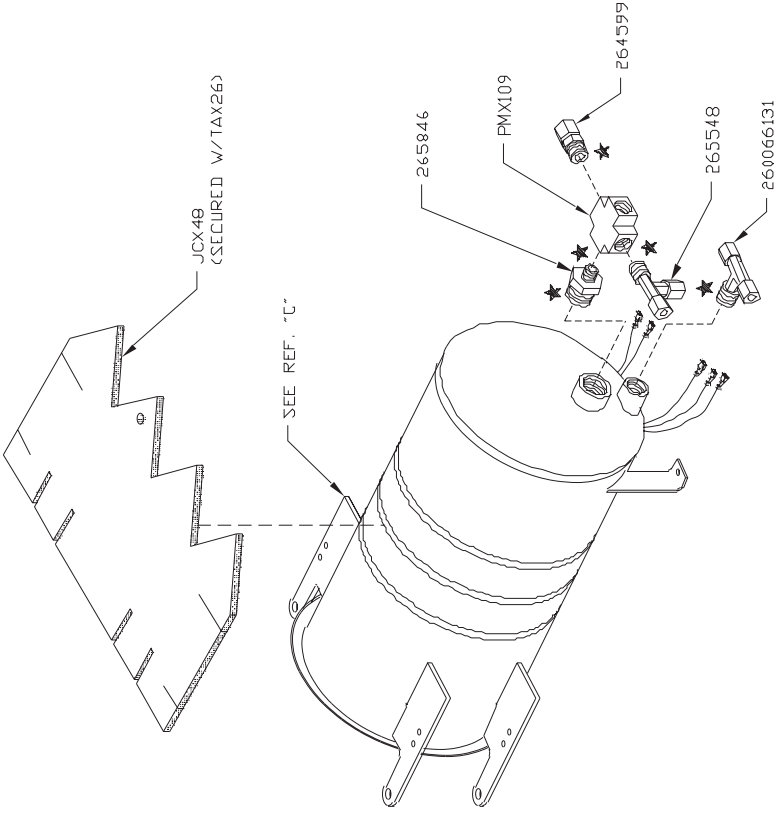



NOTES:

- ★ APPLY 260326 (SEALANT) TO THREADS PRIOR TO ASSEMBLY. (5 PLACES)

REF. "D"

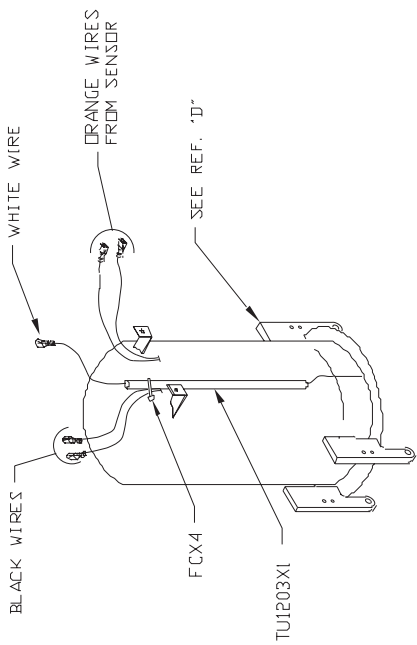

REF. "E"

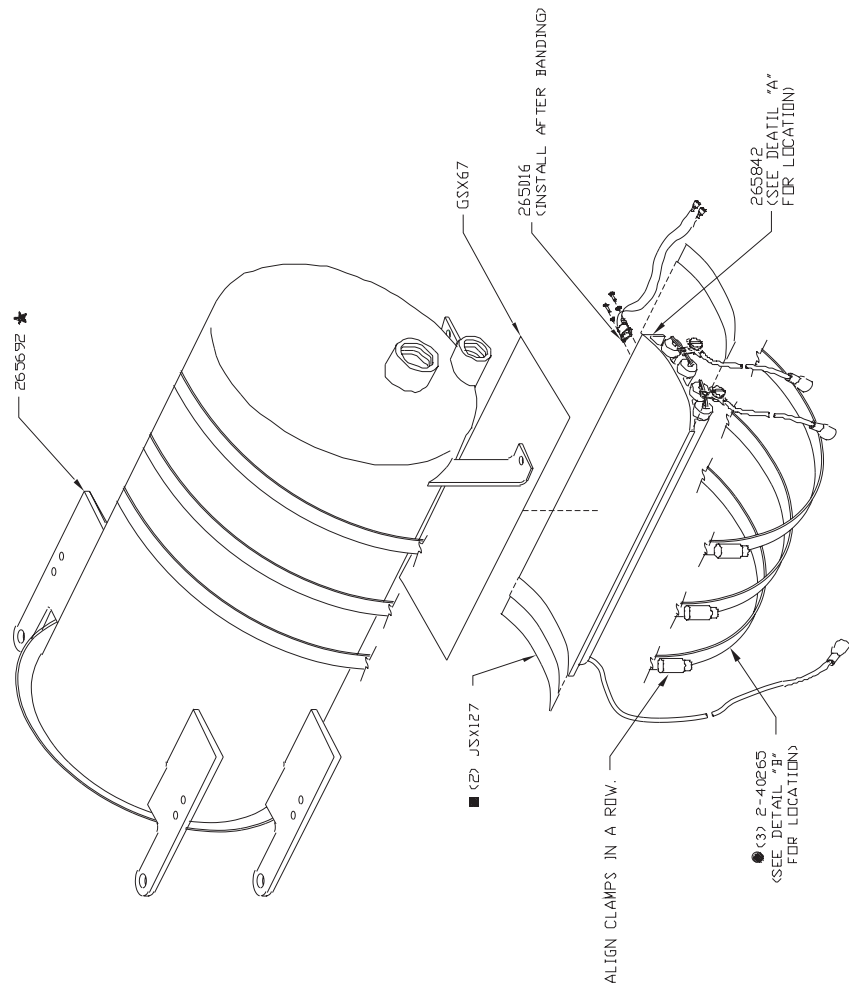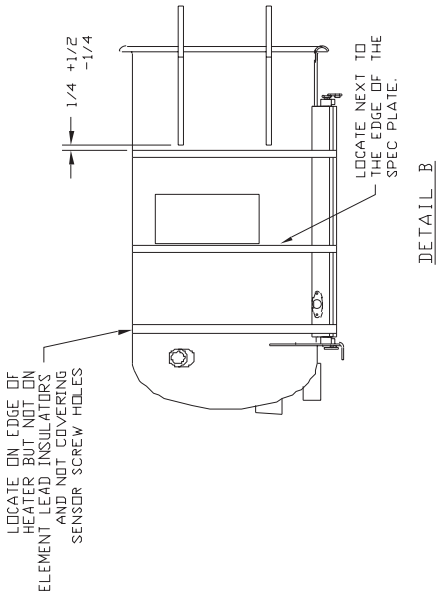



NOTES:
CENTER HEATER BETWEEN STANDOFFS AS SHOWN.


DETAIL A

- NOTES:
- BUTT MICA AGAINST ELEMENT AND OVER ELEMENT LEADS. APPLY 250016301 (ADHESIVE) BETWEEN MICA AND CERAMIC ELEMENT LEAD INSULATORS TO HOLD IN PLACE, BOTH ENDS.
 - TIGHTEN BANDS TO 43 IN-LBS.

REF. "C"

MC8

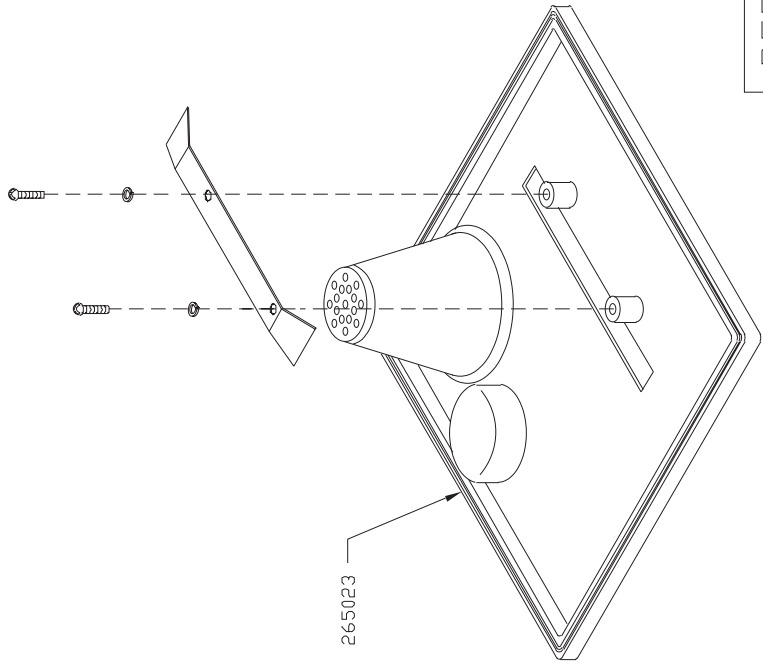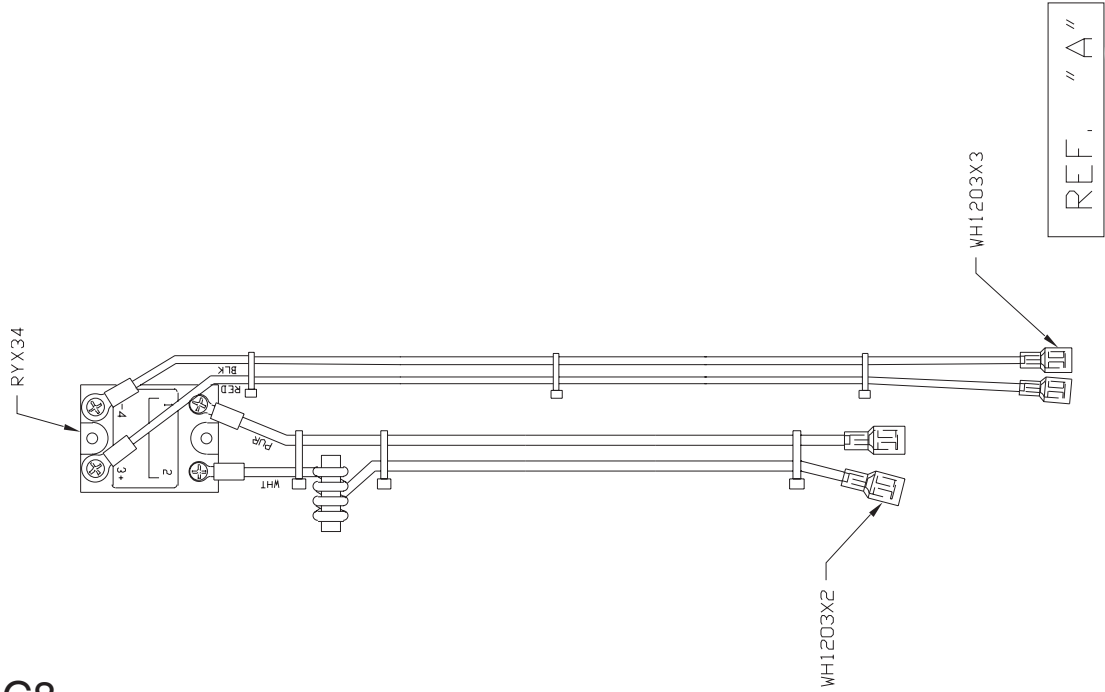

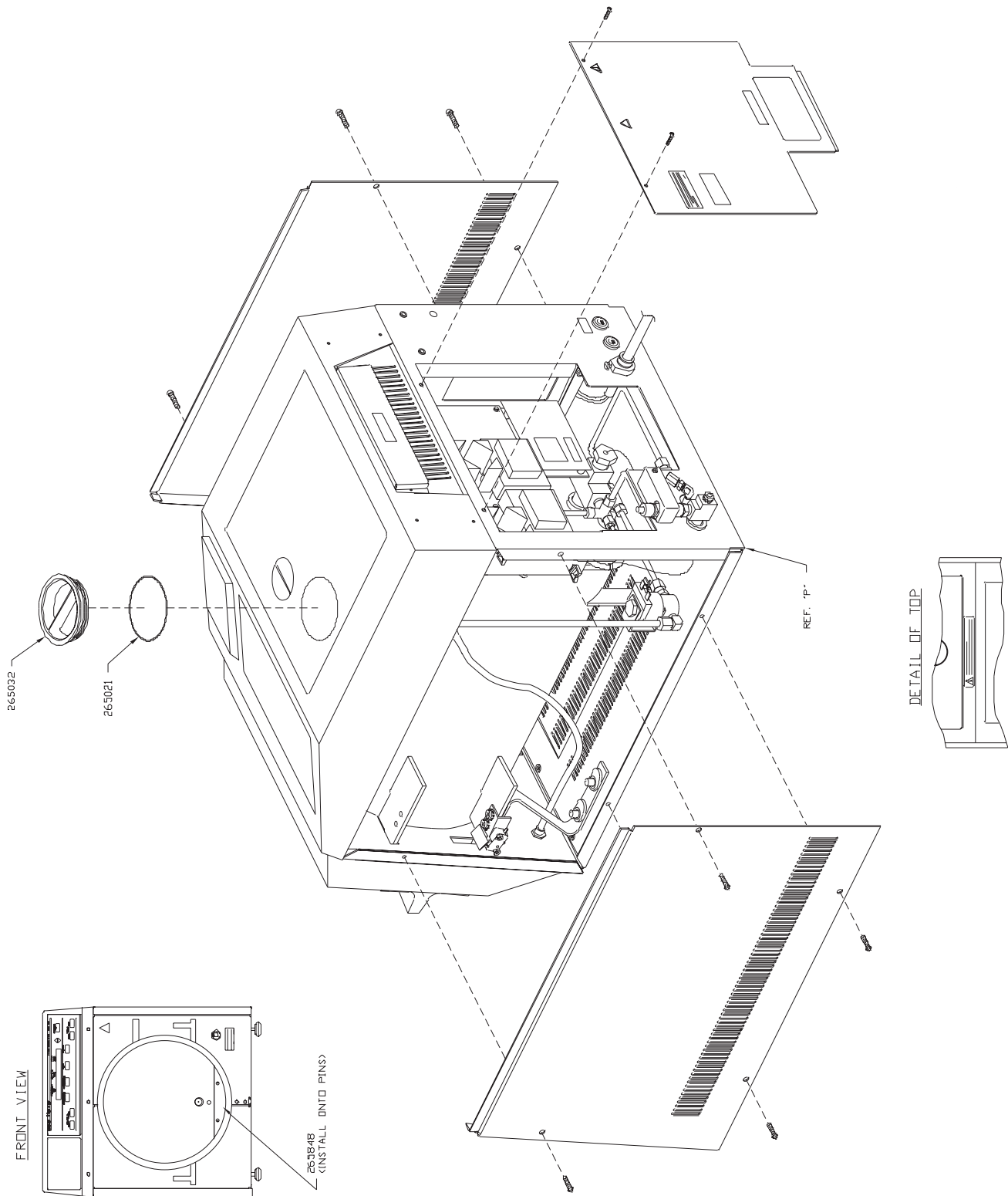

MC10

FRONT VIEW

PLACE OPEN END OF 265081 INTO HOLE.

DETAIL A

DETAIL B (DISPLAY BOARD)


DETAIL C

NOTES:

- APPLY 260326 (SEALANT) TO THREADS PRIOR TO ASSEMBLY. (1 PLACE)
- ASME PART - PROVIDE STERILIZER SERIES NUMBER WHEN ORDERING

REF. "J"


MC10

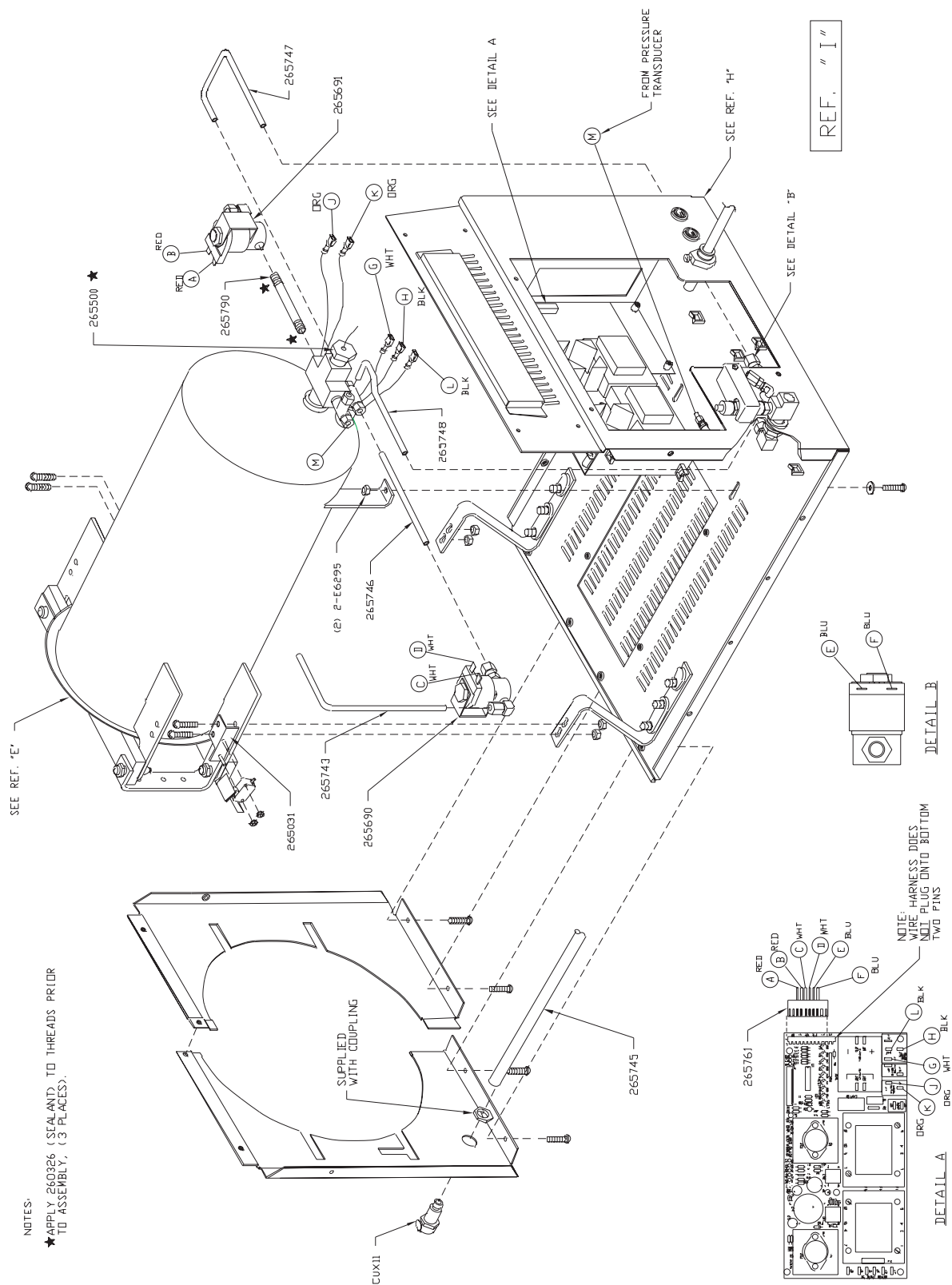

- SOME UNITS USE ONE FUSE ASSEMBLY ONLY.
- 264058 - 15 AMP FUSE - 115V UNITS.
- 264306 - 8 AMP FUSE - 230V UNITS.
- "C" WITHOUT PRINTER OPTION
- "Y" WITH PRINTER OPTION
- "B" WITHOUT PRINTER OPTION
- "Z" WITH PRINTER OPTION
- ▲ LETTER "Y" ON SINGLE FUSE UNITS.
- LETTER "Y" ON DUAL FUSE UNITS.
- ▲▲ "R" ON 115V UNITS
- "S" ON 230V UNITS
- * NOT USED ON 230V UNITS.
- ★ PART NUMBER OF WIRE.



MC10

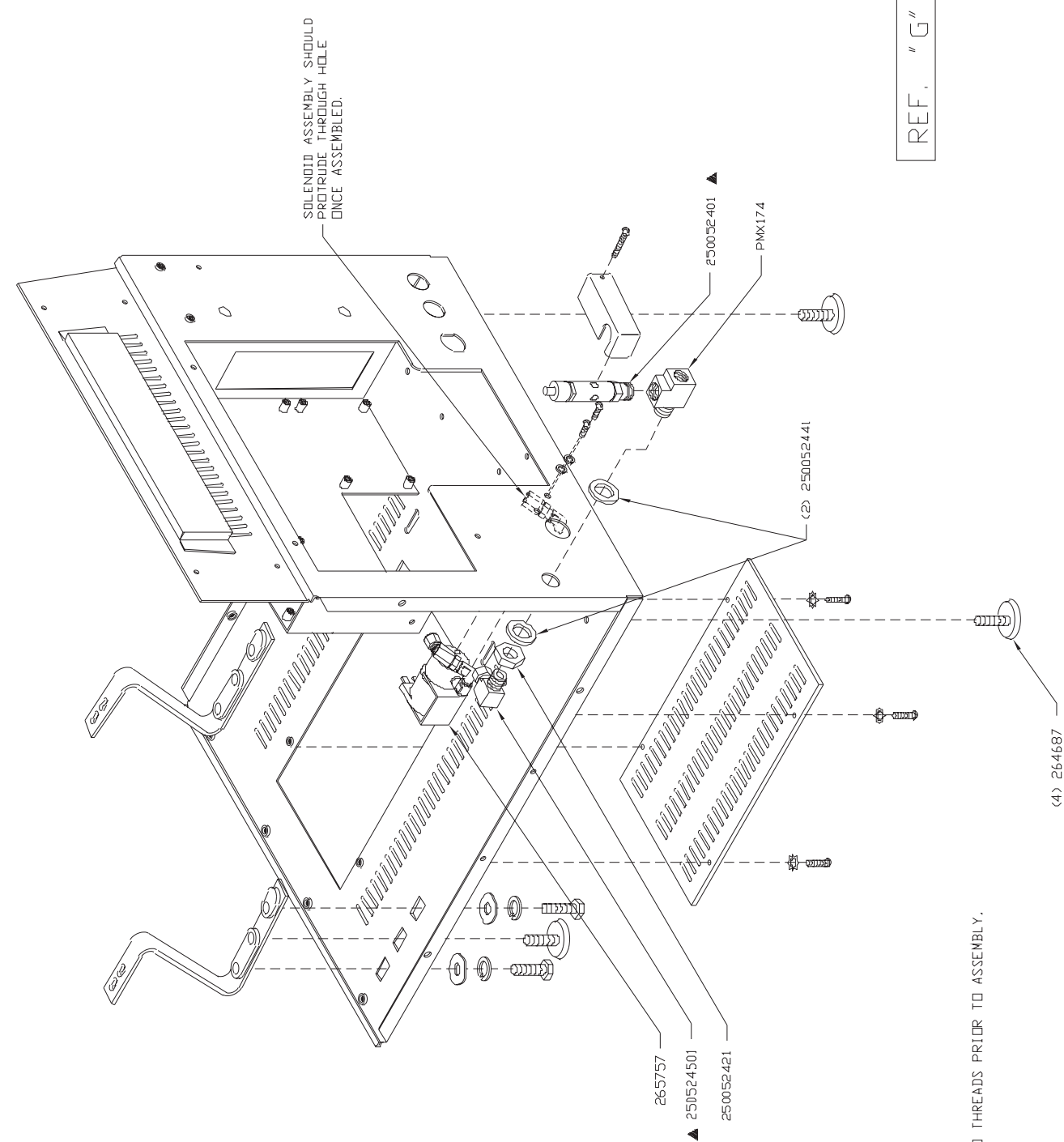


REF. "G"


NOTES

- ▲ APPLY 260326 (SEALANT) TO THREADS PRIOR TO ASSEMBLY. (2 PLACES).

MC10

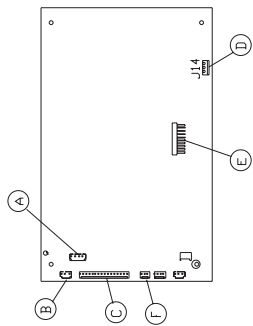


DETAIL A

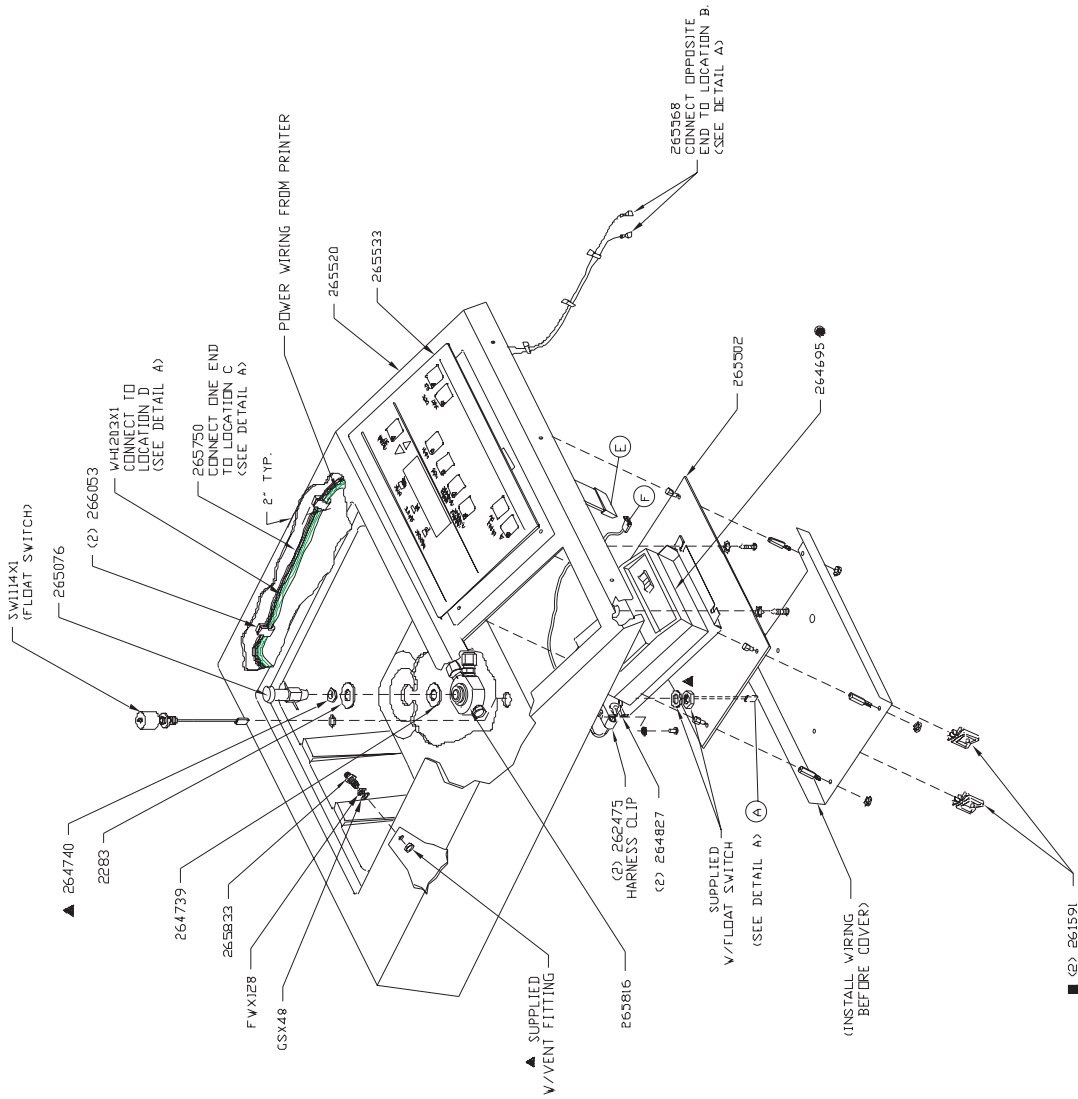

NOTES:

- ▲ TORQUE TO 12 IN. LBS. (3 PLACES)
DO NOT USE SEALER ON THIS ASSEMBLY.
- ROUTE FLOAT SWITCH AND PRINTER ASSEMBLY WIRING THROUGH BOTH CLAMPS.
- ★ INCLUDES KEYPAD AND FLOAT SWITCH.
- ONLY USED IN UNITS WITH PRINTER OPTION


REF. "F"

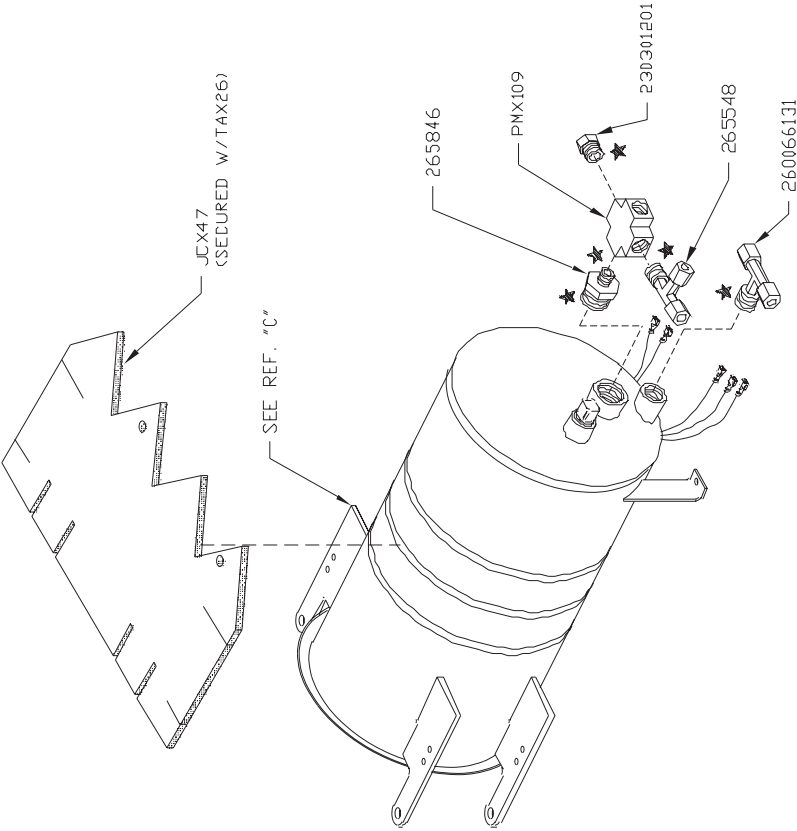

NOTES:


- ★ APPLY 260326 (LOCTITE #569) TO THREADS PRIOR TO ASSEMBLY. (5 PLACES)

REF. "D"

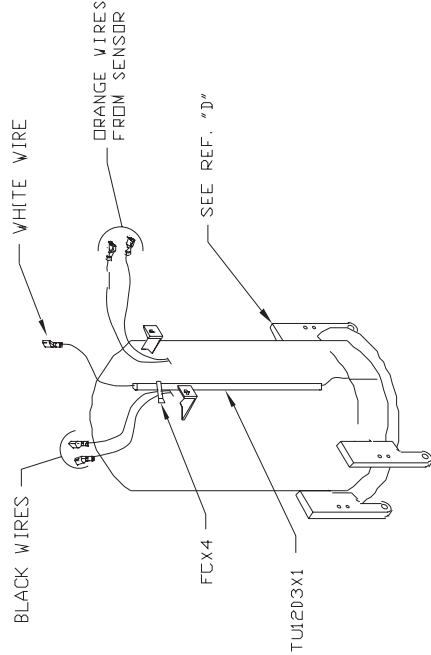

REF. "E"

MC10

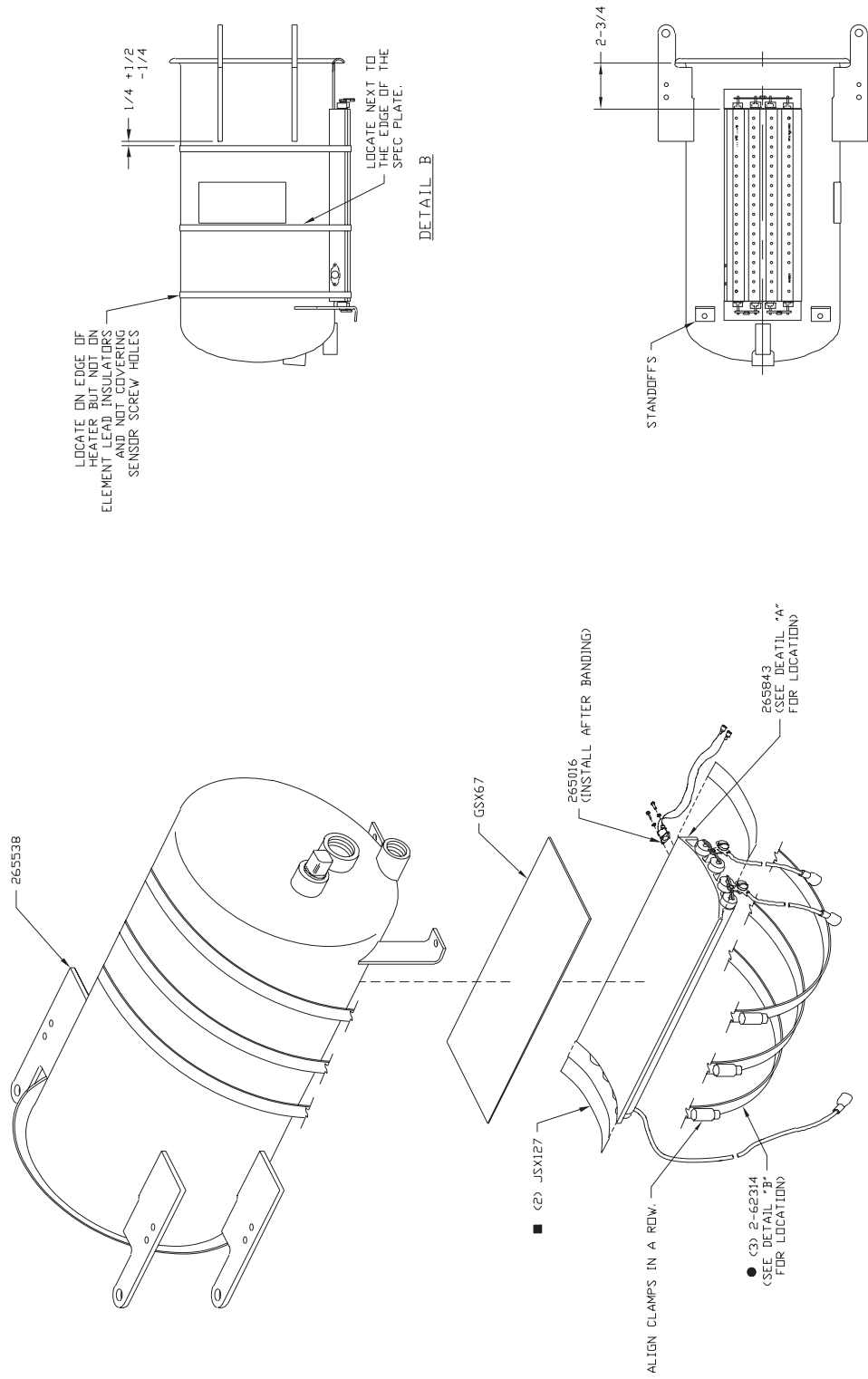


NOTES:
CENTER HEATER BETWEEN STANDOFFS AS SHOWN


- NOTES:
- BUTT MICA AGAINST ELEMENT AND OVER ELEMENT LEADS. APPLY 250016301 (ADHESIVE) BETWEEN MICA AND CERAMIC ELEMENT LEAD INSULATORS TO HOLD IN PLACE, BOTH ENDS.
 - TIGHTEN BANDS TO 45 IN-LBS.

REF. "C"

MC10

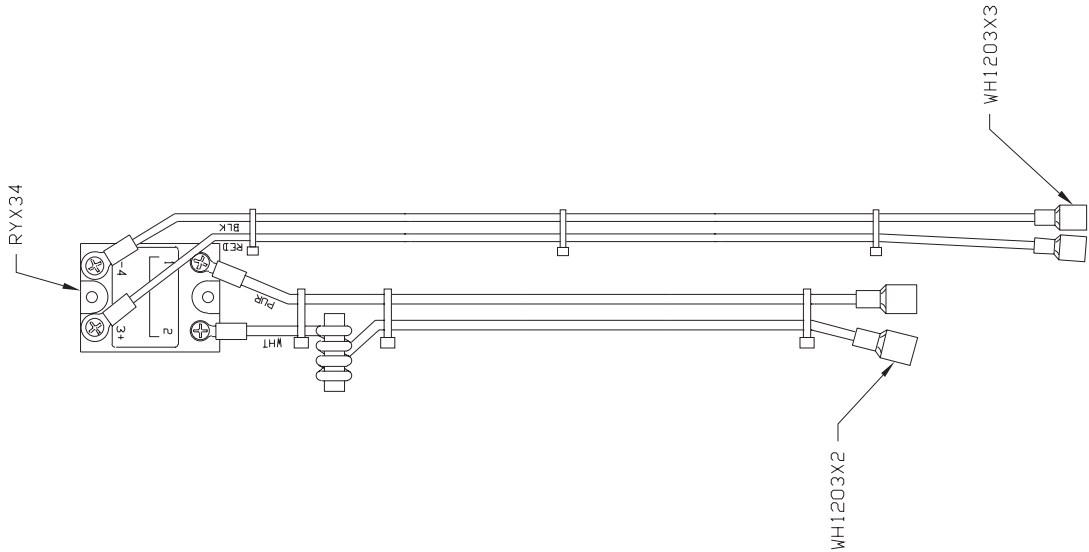


REF. "A"

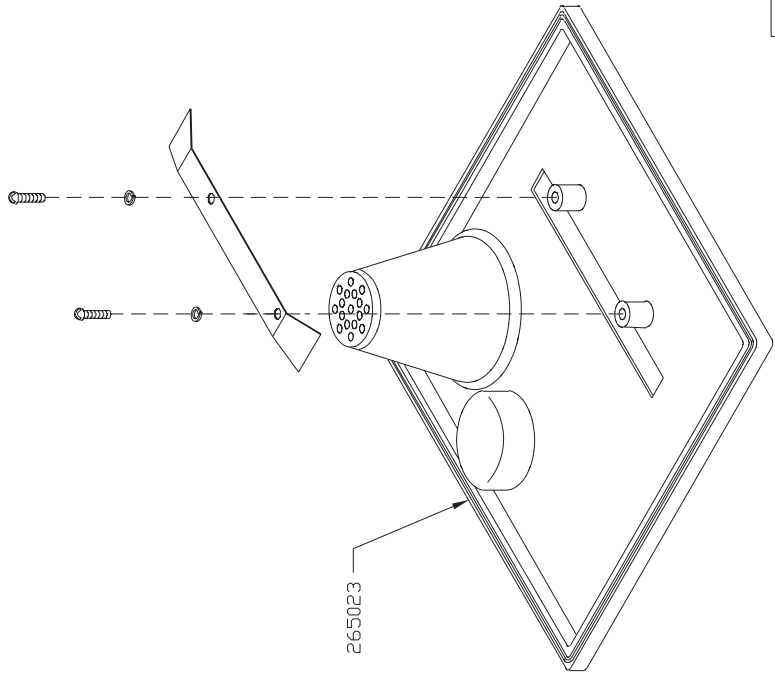

REF. "B"

Accessory Parts

PART NO.	DESCRIPTION	
267414	Drain Hose	MC8/MC10
LT1203X1	Operation Manual	MC8/MC10
LT1203X2	Service Manual	MC8/MC10
LT1203X3	Parts Manual	MC8/MC10
2—67811	Chamber Tray (Large)	MC10
265001	Chamber Tray	MC8/MC10
265002	Chamber Rack	MC8
2—513025	Chamber Rack	MC10
LS1114X1	Cleaning Pad	
AY759X1	Biological Indicators	
AY759X2	Chemical Indicators	
230300001	Chemitest Indicator Bag (Tray)	
230301001	Chemitest Indicator Bag	
265656	Non-Recirculating Option (Instrument)	
260284	Printer Paper, (6) Pack	
264116	Printer Ribbon	
SFX11	Printer Paper Support Shaft	
265870	Printer Accessory Kit (Dual Volts)	
265593	Printer Accessory Kit (115V)	
265590	Printer Accessory Kit (230V)	

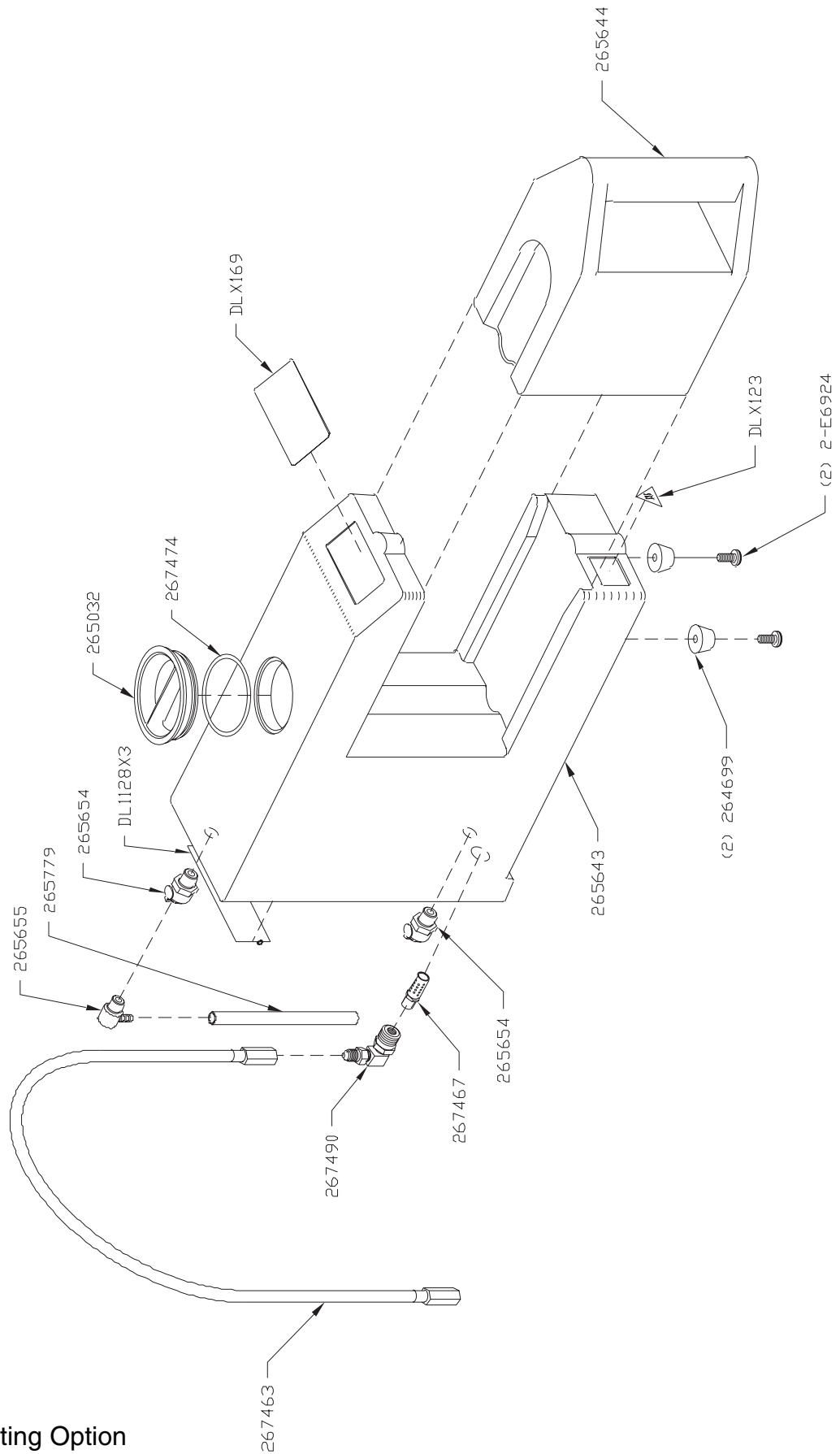

Non-Recirculating Option

Recommended Spare Parts List

<u>Qty.</u>	<u>Part No.</u>	<u>Description</u>
1	265109	Door Gasket Kit, MC8
1	2-513005	Door Gasket, MC10
1	CUX11	Drain Coupling
1	267414	Drain Hose Assembly (including Fitting)
1	265076	Filter, Reservoir
1	266058	Fuse, 15 Amp Time delay, 5mm x 20mm (115V units)
2	264306	Fuse, 8 Amp Time Delay, 5mm x 20mm (230V units)
1	260284	Printer Paper Roll (Package of 6 rolls)
1	264116	Printer Ribbon
1	265081	Screen, Chamber
1	265690	Solenoid Valve #2, Drain/Fill
1	265839	Solenoid Valve #1, Vent, MC8
1	265691	Solenoid Valve #1, Vent, MC10
1	265757	Solenoid Valve #3, Non-Recirculating
1	265030	Switch, Door Lock



Note

It is recommended that the parts listed above be maintained in stock to ensure parts are immediately available.
