


Sterilizer Main Components

SECTION VI PARTS LIST

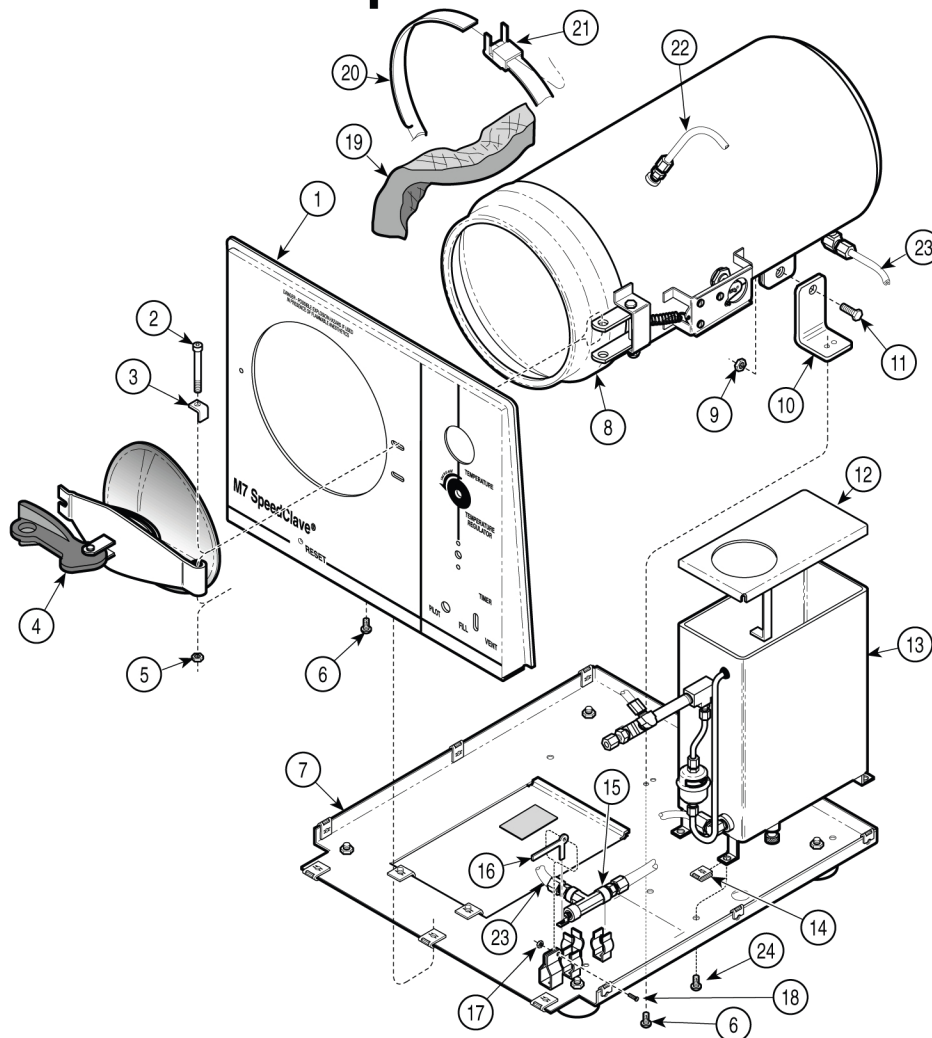

MA242702

Used on Units with Serial Numbers:
CP1062 thru CP1149 CR1155 thru CR1511 CS1976 thru CS7753
RB7987 thru RB17992 DR1015 thru DR1267

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1		Panel (Refer to "Front Panel Components" Elsewhere)	Ref	9	MZY100027	Nut	1
2	MZY100019	Screw	1	10	MZY100033	Chamber Leg Bracket	1
3	MZY100020	Door Stop	1	11	MZY100029	Screw	1
4		Door Assembly (Refer to "Door Assembly" Elsewhere)	Ref	12	MZY100030	Tank Cover	1
5	MZY100021	Torque Nut	1	13		Tank Assembly (Refer to "Tank Assembly" Elsewhere)	Ref
6	MZY100007	Screw	5	14	MZY100031	Clip	4
7		Base Assembly (Refer to "Base Components" Elsewhere)	Ref	15		Vent Valve Components (Refer to "Valve Assembly" Elsewhere)	Ref
8	MZY100023	Shell Assembly [M7-001 & M7-004] (Refer to "Chamber Components {Side or Lower}" Elsewhere)	1	16	MZY100032	Lever	1
	MZY100024	Shell Assembly [M7-005] (Refer to "Chamber Components {Side or Lower}" Elsewhere)	1	17	MZY100021	Nut	1
	MZY100025	Shell Assembly [M7-002] (Refer to "Chamber Components {Side or Lower}" Elsewhere)	1	18	MZY100034	Screw	1
	MZY100026	Shell Assembly [M7-003] (Refer to "Chamber Components {Side or Lower}" Elsewhere)	1	19	MZY100035	Insulation	1
				20	MZY100036	Banding Strap (shown)	2
				21	MZY100037	Banding Strap End	1
				22	MZY100038	Banding Strap Clip	2
				23	MZY100039	Tubing	1
				24	MZY100040	Tubing	1
					MZY100042	Screw	4

Always Specify Model & Serial Number