


Front Panel Components

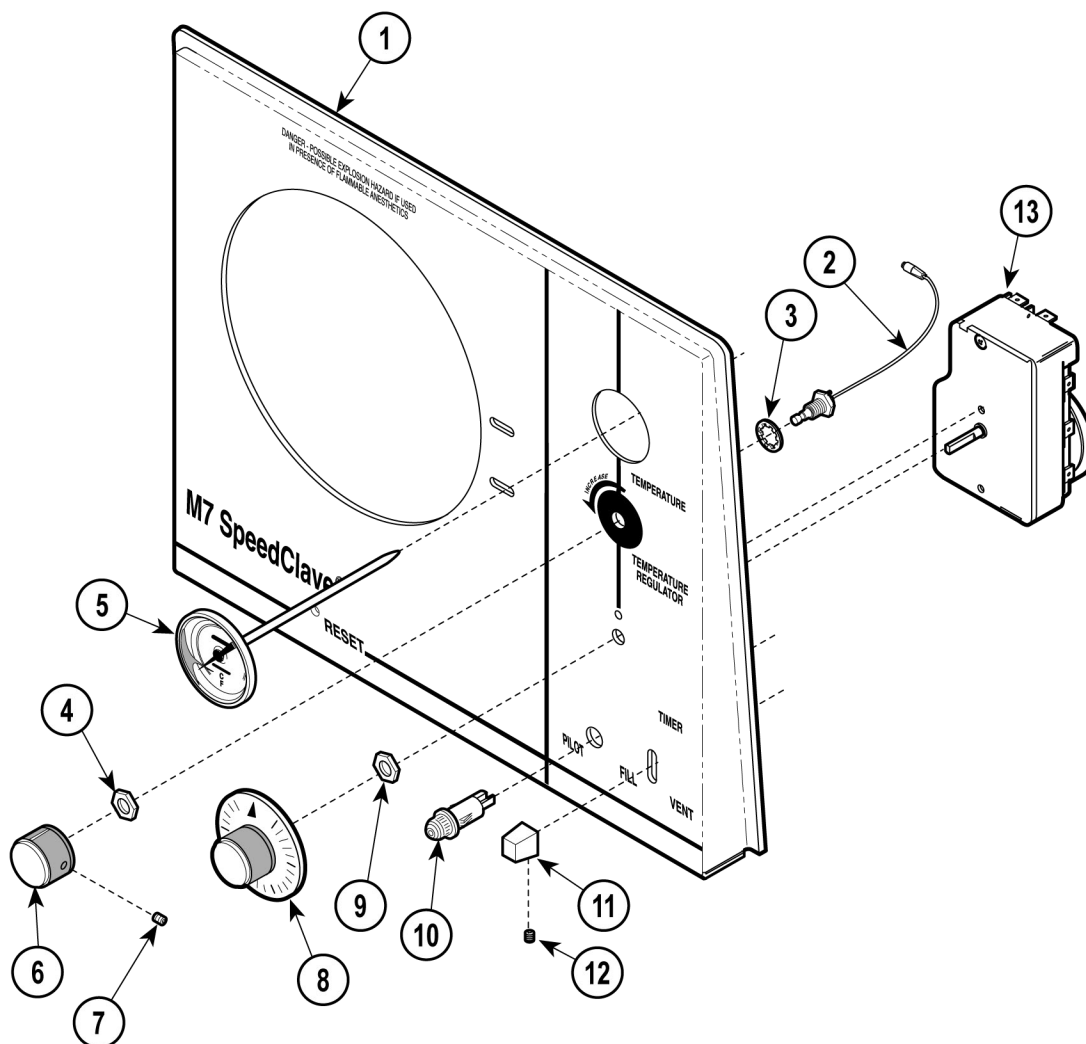

Used on units with Serial Number RB8000 thru RB8106

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1	MZY100047	Ritter Panel (M7-001)	1	8	MZY100054	Timer Knob	1
	MZY100066	Midmark Panel (M7-002, M7-003, M7-004)	1	9		Nut	Ref
2	MZY100049	Flexible Shaft Assembly (Includes Items 3 and 4)	1	10	MZY100055	Pilot Light (M7-001, M7-003, M7-004)	1
3	• MZY100050	• Lockwasher	1		MZY100056	Pilot Light (M7-002)	1
4	• MZY100051	• Nut	1	11	MZY100057	Fill/Vent Knob Assembly (Includes Item 12)	1
5	MZY100052	Thermometer	1	12	• MZY100058	• Setscrew	1
6	MZY100053	Control Knob (Includes Items 7)	1	13	MZY100059	Timer (M7-001, M7-003, M7-004)	1
7	•	• Setscrew	1		MZY100060	Timer (M7-002)	1

Always Specify Model & Serial Number