


Front Panel Components

SECTION VI PARTS LIST

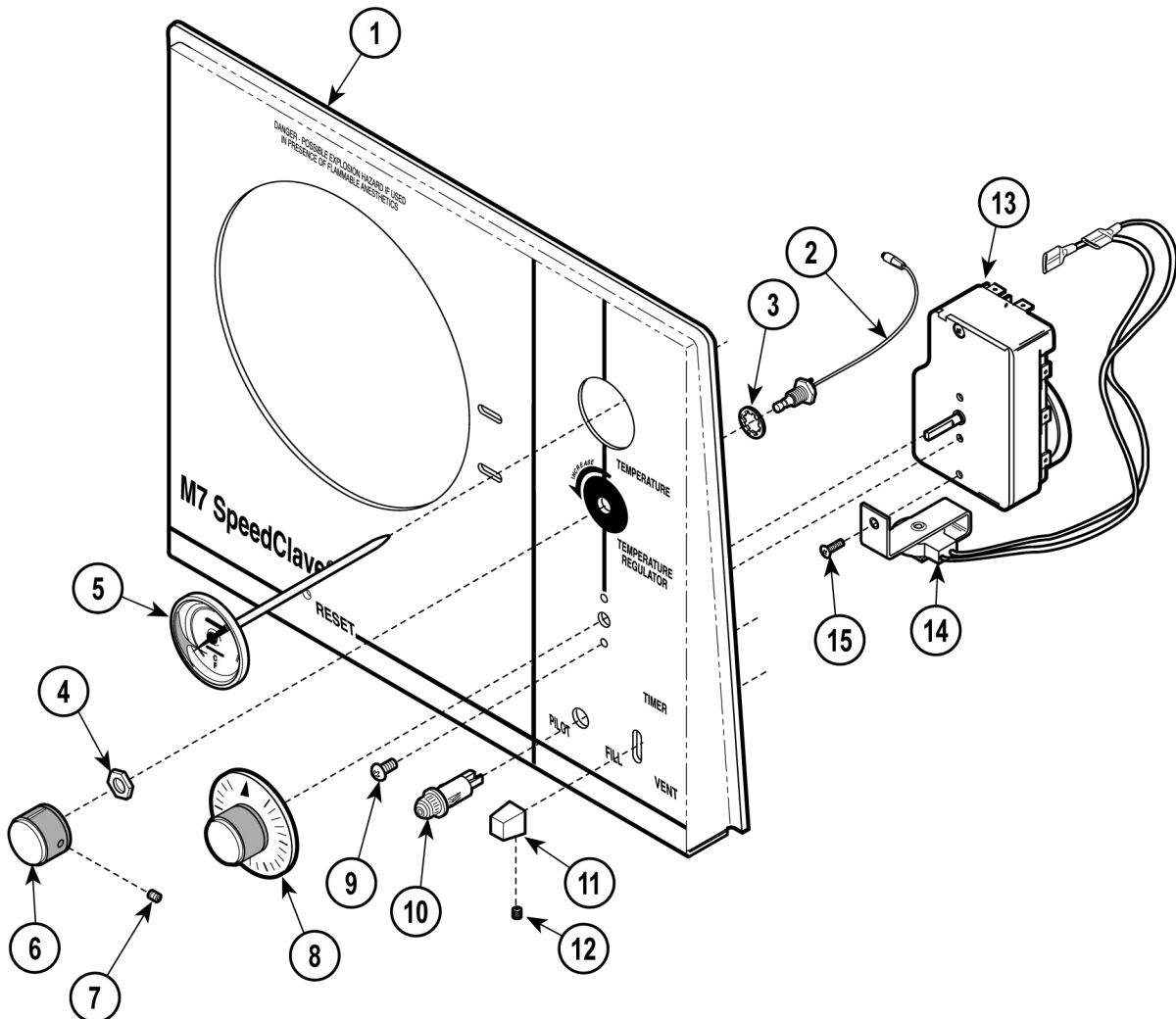

MA241101

Used on Units with Serial Numbers:
CP1070 thru CP1091 CR1126 thru CR1259 CS1695 thru CS3488
RB8107 thru RB11004

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1	MZY100061	Ritter Panel (M7-001, M7-005)	1	9	MZY100063	Screw	2
	MZY100062	Midmark Panel (M7-002, M7-003, M7-004)	1	10	MZY100055	Pilot Light (M7-001, M7-003, M7-004, M7-005)	1
2	MZY100049	Flexible Shaft Assembly (Includes Item 3 and 4)	1		MZY100056	Pilot Light (M7-002)	1
3	• MZY100050	• Lockwasher	1	11	MZY100057	Fill / Vent Knob Assy. (Includes Item 12)	1
4	• MZY100051	• Nut	1	12	• MZY100058	• Setscrew	1
5	MZY100052	Thermometer	1	13	MZY100059	Timer (M7-001, M7-003, M7-004, M7-005) [Incl. Items 14 and 15]	1
6	MZY100053	Control Knob (Includes Item 7)	1		MZY100060	Timer (M7-002) [Incl. Items 14 and 15]	1
7	•	• Setscrew	1	14	• MZY100064	• Timer Buzzer (120 volt)	1
8	MZY100054	Timer Knob	1		• MZY100065	• Timer Buzzer (230 volt)	1
				15	• MZY100063	• Screw	1

Always Specify Model & Serial Number