


Front Panel Components

SECTION VI PARTS LIST

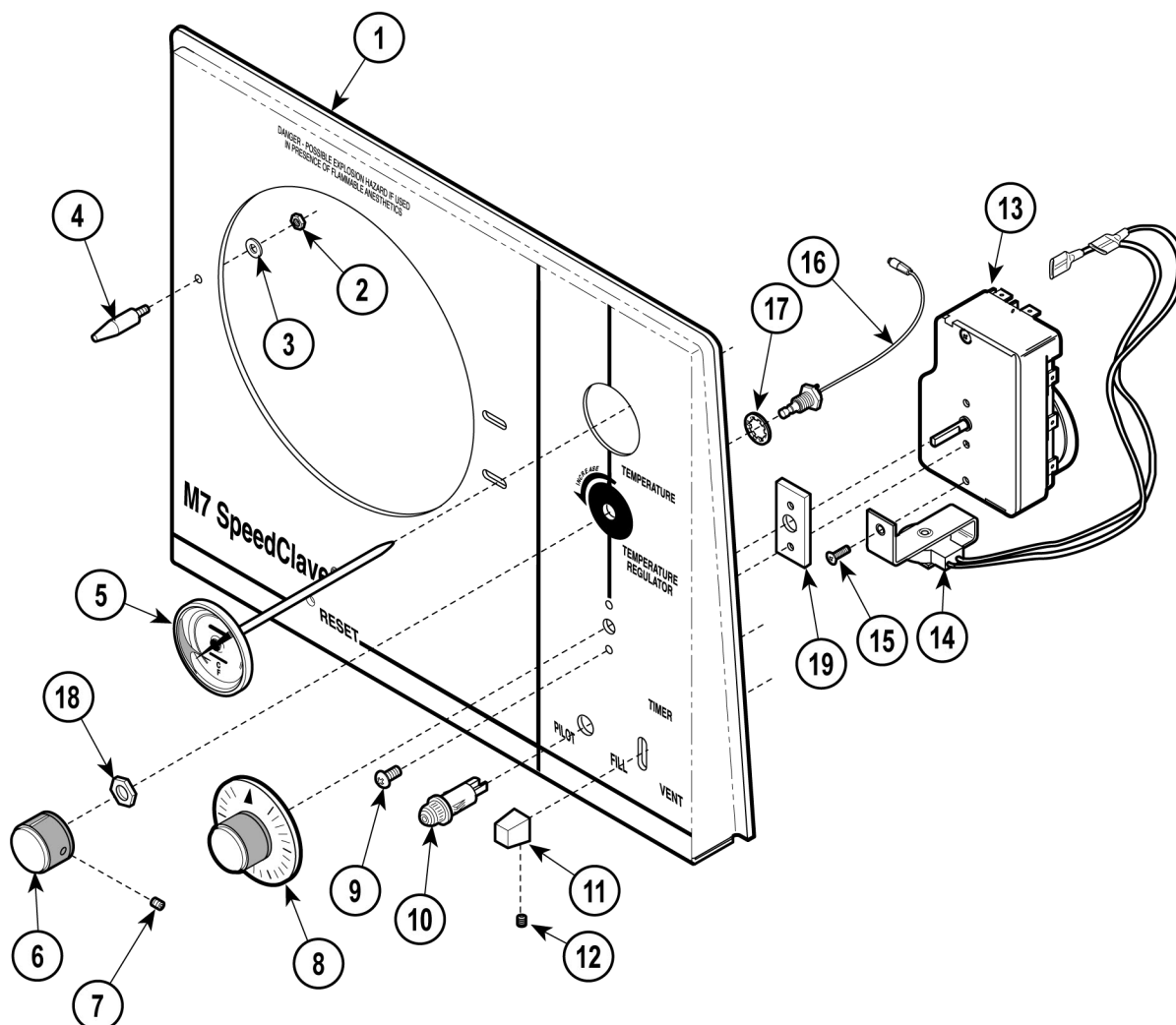

MA241103

Used on Units with Serial Numbers RB11005 thru RB18474, CP1092 thru CP1149, DR1015 thru DR1267, CR-1260 thru CR1724, CS3489 thru CS8328, FM1000 thru FM1545 and FN1000 thru FN1309

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1	MZY100067 MZY100068	Ritter Panel (M7-001)	1	11	MZY100056 MZY100073	Pilot Light (M7-002)	1
		Midmark Panel (M7-002, M7-003, M7-004)	1	12	• MZY100058	Fill / Vent Knob Assembly (Includes Item 12)	1
		Special Panel (M7-005, M7-006)	1	13	• MZY100059	• Setscrew	1
2	MZY100070	Nut	1			Timer (M7-001, M7-003, M7-004, M7-005) [Incl. Items 14 and 15]	1
3	MZY100071	Lockwasher	1	14	• MZY100064	Timer (M7-002) [Incl. Items 14 and 15] ..	1
4	MZY100072	Alignment Pin	1		• MZY100065	• Timer Buzzer (120 volt)	1
5	MZY100052	Thermometer	1		• MZY100065	• Timer Buzzer (230 volt)	1
6	MZY100053	Control Knob (Includes Item 7)	1	15	• MZY100063	• Screw	1
7	•	• Setscrew	1	16	MZY100049	Flexible Shaft Assembly (Includes Item 17 and 18)	1
8	MZY100054	Timer Knob	1	17	• MZY100050	• Lockwasher	1
9	MZY100063	Screw	2	18	• MZY100051	• Nut	1
10	MZY100055	Pilot Light (M7-001, M7-003, M7-004, M7-005)	1	19	MZY100074	Spacer	1

Always Specify Model & Serial Number