


Door Assembly

SECTION VI PARTS LIST

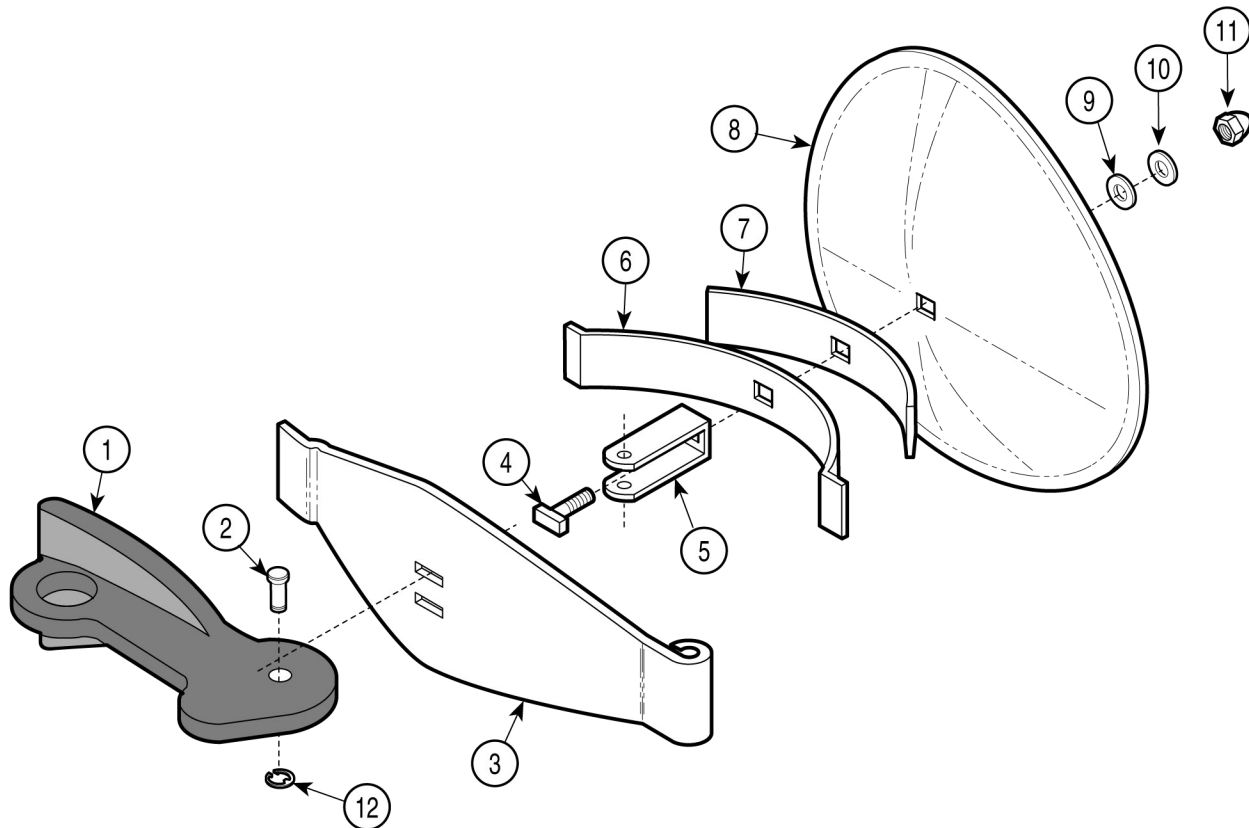

MA240700

Used on Units with Serial Numbers:
CP1072 thru CP1091 CR1226 thru CR1259 CS3104 thru CS3488
RB10146 thru RB11004

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	MZY100083	Door Assembly (Includes Items 1 thru 12)	1	6	MZY100089	Long Spring	1
1	MZY100084	Handle (Apply Valve Lubricant #064-0002-01)	1	7	MZY100090	Short Spring	1
2	MZY100097	Pin	1	8	MZY100091	Door	1
3	MZY100098	Door Cross Arm	1	9	MZY100092	Gasket	1
4	MZY100087	"T" Bolt	1	10	MZY100093	Washer	1
5	MZY100088	"U" Bracket	1	11	MZY100094	Cap Nut (Apply Loctite #042-0024-02)	1
				12	MZY100100	Retaining Ring	1

Always Specify Model & Serial Number