


Door Assembly

SECTION VI PARTS LIST

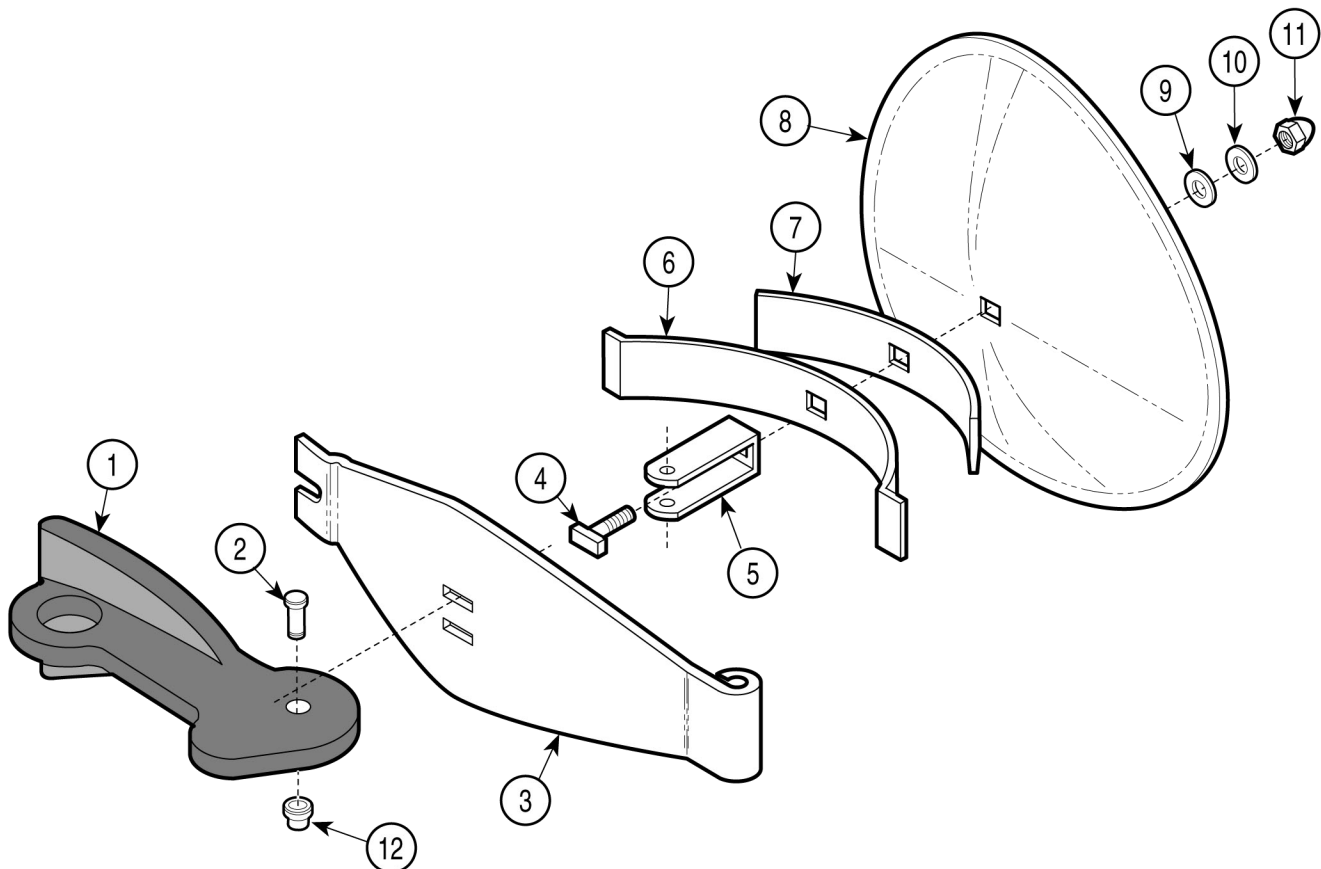

MA240703

Used on Units with Serial Numbers RB11005, CP1092, DR1015, CR1260, CS3489, FM1000 and FN1000 thru Present

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	MZY100083	Door Assembly (Includes Items 1 thru 12)	1	6	MZY100089	• Long Spring	1
1	• MZY100084	• Handle (Apply Valve Lubricant #064-0002-01)	1	7	• MZY100090	• Short Spring	1
2	• MZY100085	• Clevis Pin	1	8	• MZY100091	• Door	1
3	• MZY100086	• Door Cross Arm	1	9	• MZY100092	• Gasket	1
4	• MZY100087	• "T" Bolt	1	10	• MZY100093	• Washer	1
5	• MZY100088	• "U" Bracket	1	11	• MZY100094	• Cap Nut (Apply Loctite #042-0024-02) .	1
				12	• MZY100095	• Push Nut	1

Always Specify Model & Serial Number