


Base Components

SECTION VI PARTS LIST

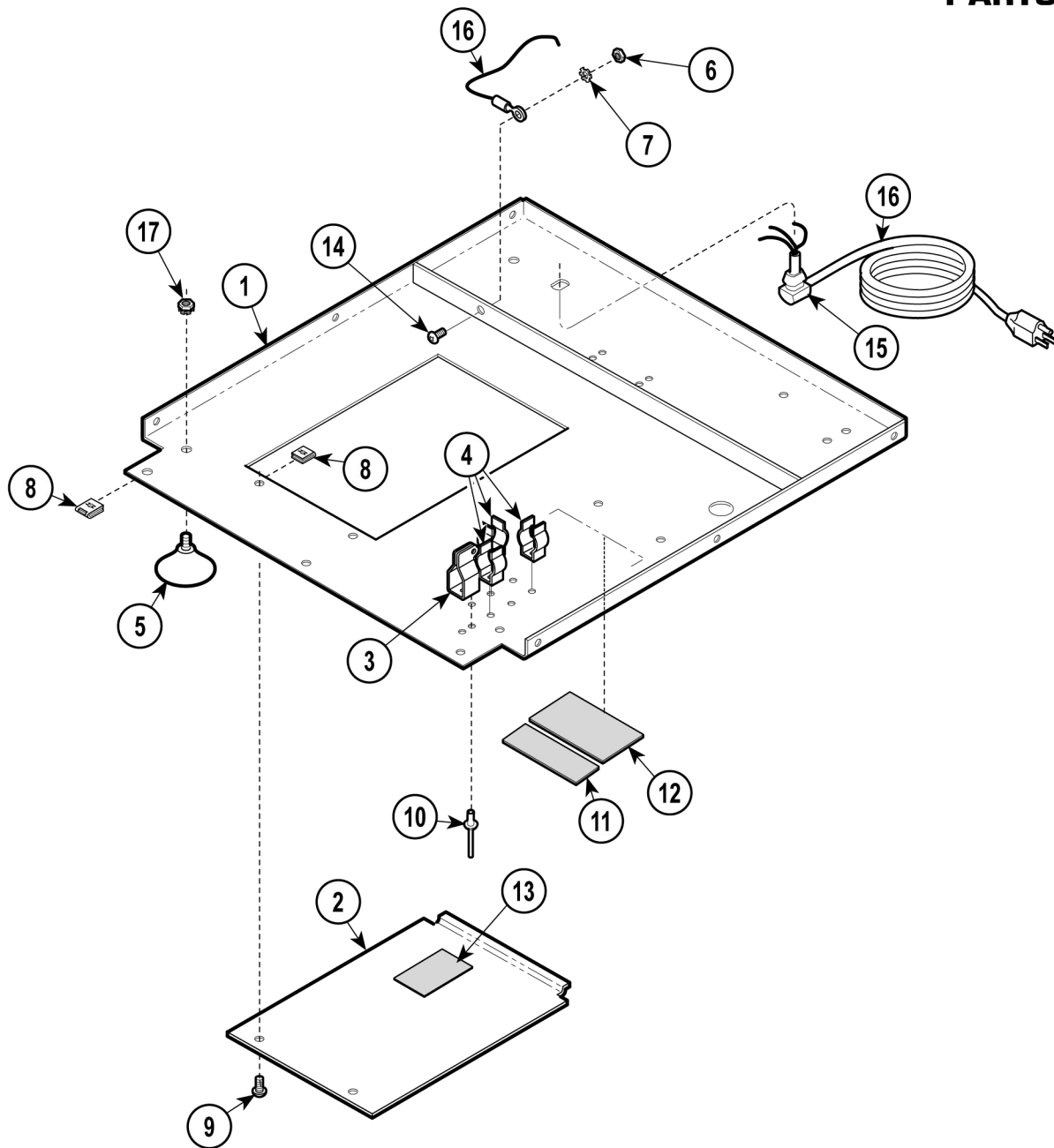

MA241201

Used on Units with Serial Numbers:
CP1000 thru CP1061 CR1000 thru CR1154 CS1000 thru CS1975
RB1000 thru RB7986 DR1000 thru DR1014

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1	MZY100101	Base	1	12	MZY100109	Serial Number Tag	Ref
2	MZY100102	Inspection Cover	1	13	MZY100041	Warning Label	1
3	MZY100103	Vent Lever Bracket	1	14	MZY100071	Screw	1
4	MZY100104	Clip	3	15	MZY100110	Lockwasher	1
5	MZY100105	Foot	4	15	MZY100111	Strain Relief	1
6	MZY100070	Nut	4	16	MZY100112	Cord (M7-001, M7-003, M7-004, M7-005)	1
7	MZY100106	Lockwasher	1	16	MZY100113	Cord (M7-002)	1
8	MZY100031	Clip	11	17	MZY100113	Keps Nut	4
9	MZY100022	Screw	2				
10	MZY100107	Poprivet	5				
11	MZY100108	U.L. Label	Ref				

Always Specify Model & Serial Number