


Base Components

SECTION VI PARTS LIST

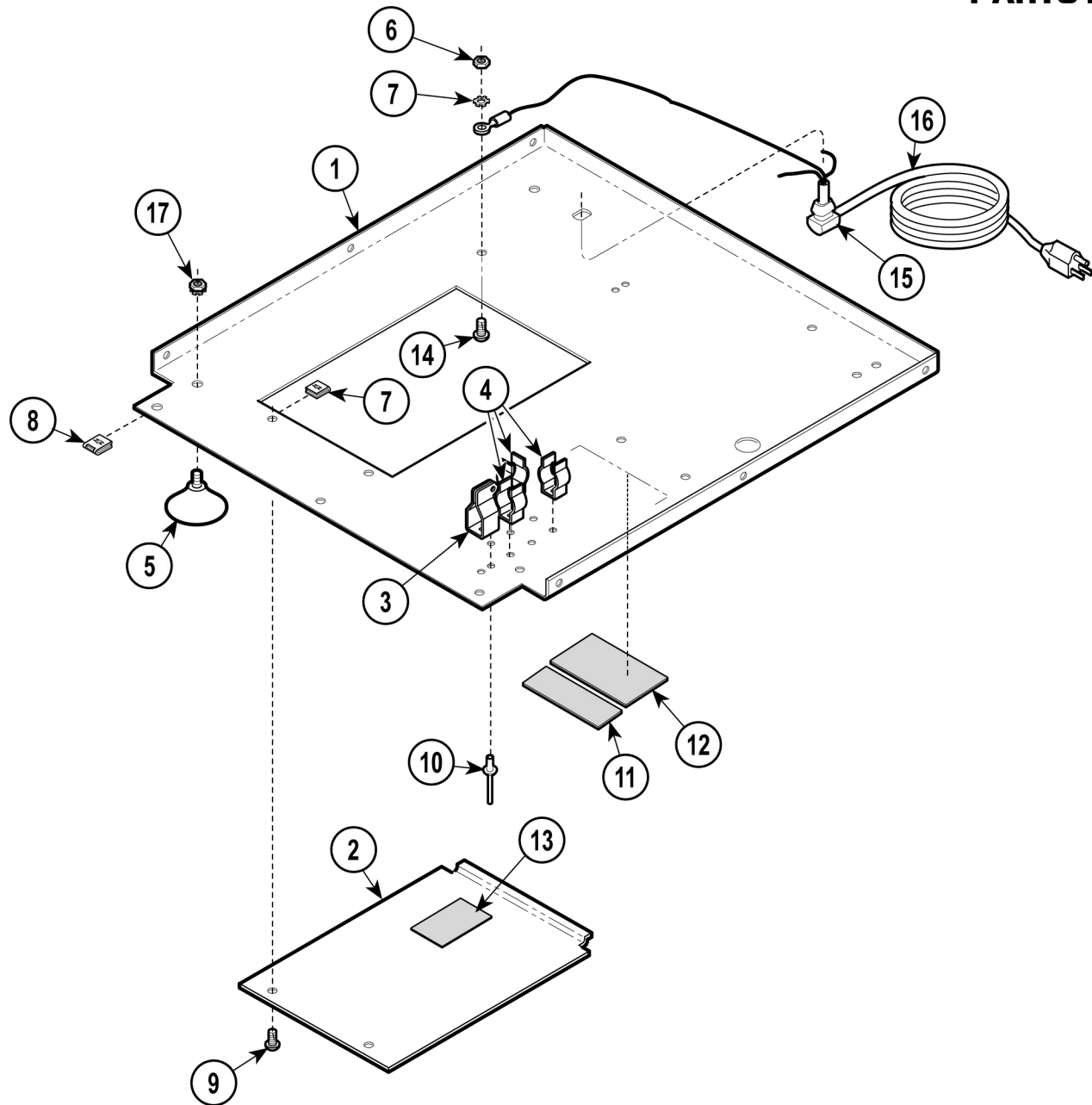

MA241205

Used on Units with Serial Numbers CR1511 thru CR1685, FM1000 thru FM1425 and FN1000 thru FN1229

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1	MZY100116	Base	1	13	MZY100115	Wiring Diagram Label (M7-001, M7-003 & M7-004)	1
2	MZY100102	Inspection Cover	1		MZY100117	Wiring Diagram Label (M7-002)	1
3	MZY100103	Vent Lever Bracket	1		MZY100118	Wiring Diagram Label (M7-005)	1
4	MZY100104	Clip	3		MZY100119	Wiring Diagram Label (M7-006)	1
5	MZY100105	Foot	4	14	MZY100041	Screw	1
6	MZY100070	Nut	4	15	MZY100110	Strain Relief	1
7	MZY100071	Lockwasher	5	16	MZY100111	Cord (M7-001, M7-003, M7-004, M7-005)	1
8	MZY100031	Clip	11		MZY100120	Cord (M7-002, M7-006)	1
9	MZY100007	Screw	2	17	MZY100113	Keps Nut	4
10	MZY100031	Poprivet	5				
11	MZY100107	U.L. Label	Ref				
12	MZY100108	Serial Number Tag	Ref				

Always Specify Model & Serial Number