


Chamber Components - Side

SECTION VI PARTS LIST

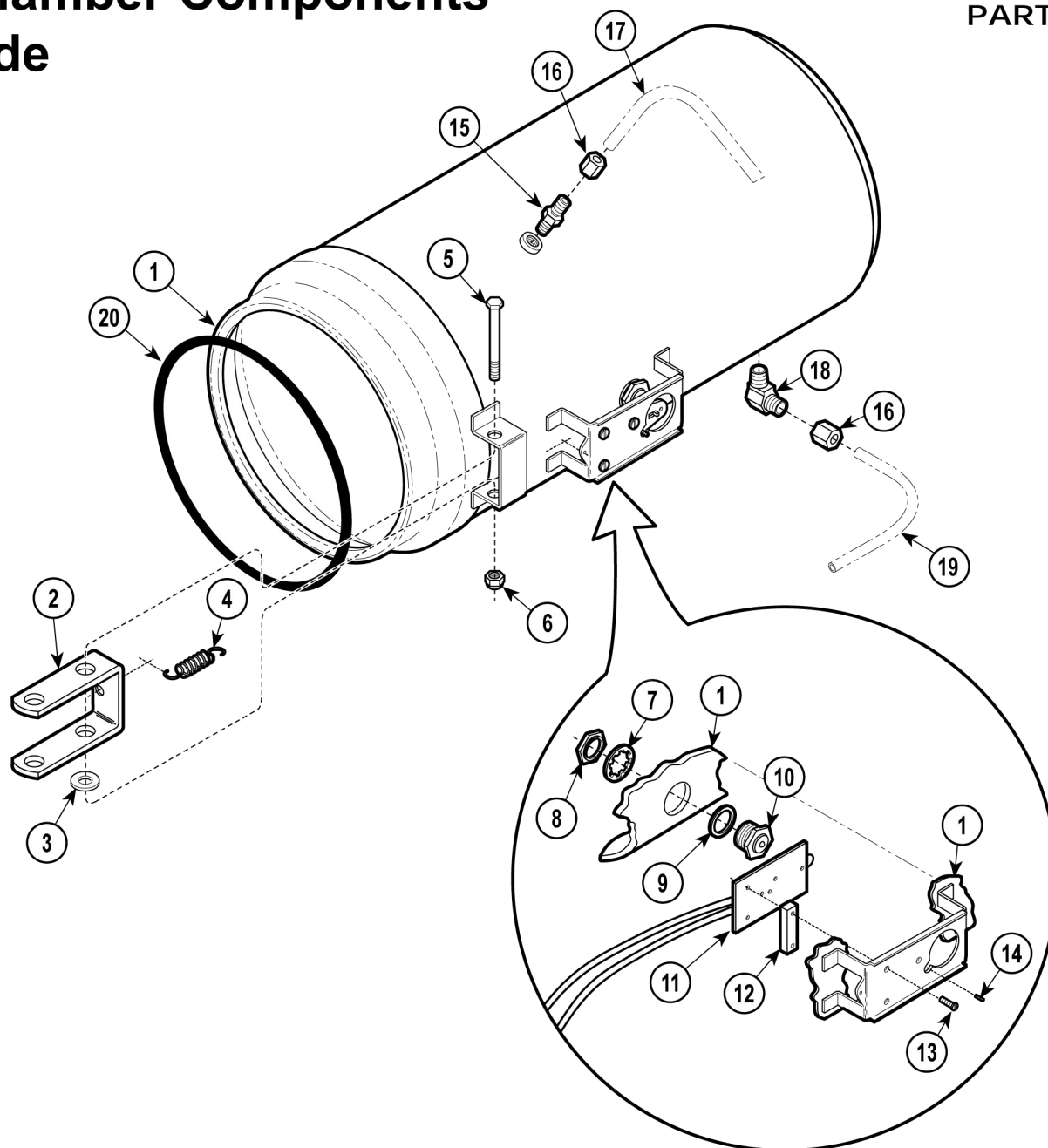

MA240903

Used on Units with Serial Numbers RB1000, CP1000, CR1000 DR1000, CS1000, CS1000, FM1000 and FN1000 thru Present

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1	MZY100121	Shell (Includes Compression Fittings) ...	1	12	MZY100132	Relay Spacer (M7-001, M7-003, M7-004, M7-005)	1
2	MZY100122	Small Hinge	1	13	MZY100133	Screw	3
3	MZY100123	Washer	1	14	MZY100134	Set Screw	1
4	MZY100124	Spring	1	15	MZY100135	Fitting	1
5	MZY100125	Screw	1	16	MZY100136	Compression Nut	1
6	MZY100027	Nut	1	17		Tubing	Ref
7	MZY100126	Lockwasher	1	18	MZY100137	Fitting	1
8	MZY100127	Nut	1	19		Tubing	Ref
9	MZY100128	Gasket	1	20	MZY100138	Door Gasket	1
10	MZY100129	Diaphragm Cup	1				
11	MZY100130	Relay (M7-001, M7-003, M7-004, M7-005)[Shown]	1				
	MZY100131	Relay (M7-002 & M7-006)	1				

Always Specify Model & Serial Number