


Chamber Components - Lower

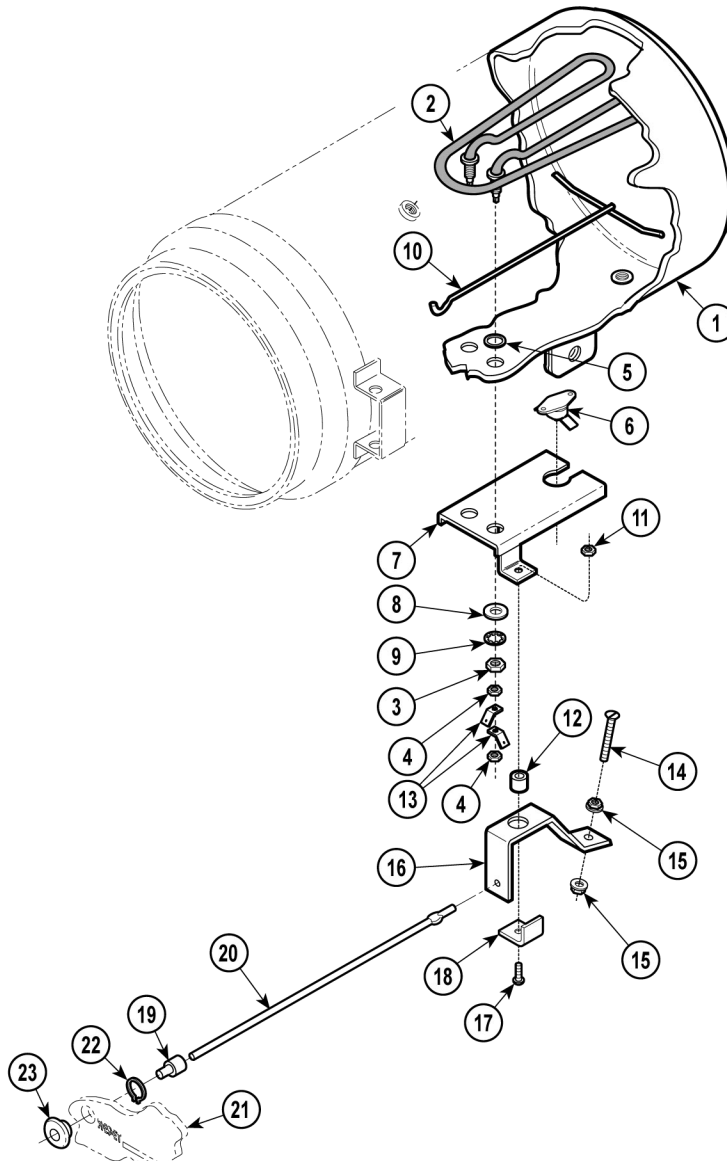

MA241001

Used on Units with Serial Numbers:
CP1000 thru CP1091 CR1000 thru CR1259 CS1000 thru CS3488
RB1000 thru RB11004 DR1000 thru DR1014

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1		Chamber (Refer to "Chamber Components - Side" Elsewhere)	Ref	13	MZY100150	Terminal	4
2	MZY100139	Heater (M7-001, M7-004, M7-005)	1		MZY100151	Rocker Assembly (Includes Item 12, Item 13 and Item 14)	1
	MZY100140	Heater (M7-002) [Includes Items 3]	1	14	MZY100152	Screw	1
	MZY100141	Heater (M7-003) [Includes Items 3]	1	15	MZY100153	Locknut	2
3	MZY100142	Nut	2	16	MZY100154	Rocker Bracket	1
4		Nut	4	17	MZY100041	Screw	1
5	MZY100143	Gasket	2	18	MZY100155	Bracket	1
6	MZY100144	Thermostat	1		MZY100156	Reset Button Kit	1
7	MZY100145	Bracket	1	19	MZY100157	Reset Button	1
8	MZY100146	Washer	1	20	MZY100158	Rod	1
9	MZY100147	Lockwasher	2	21		Front Panel (Refer to "Front Panel Components" Elsewhere)	Ref
10	MZY100148	Spacer	1	22	MZY100159	Retaining Ring	1
11	MZY100021	Nut	1	23	MZY100160	Reset Button Guide	1
12	MZY100149	Spacer	1				

Always Specify Model & Serial Number