


Chamber Components - Lower

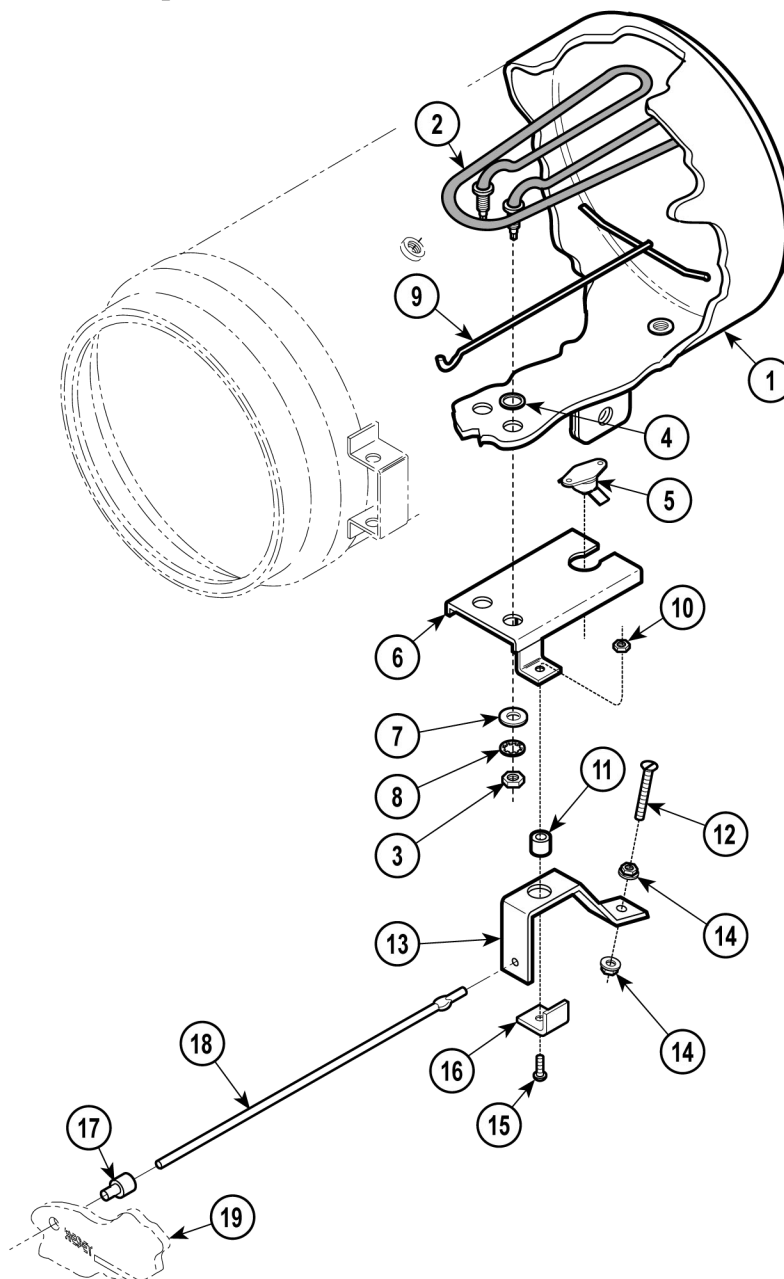

MA241000

Used on Units with Serial Numbers: CS3489 thru CS5296 RB11005 thru RB14564

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
1		Chamber (Refer to "Chamber Components - Side" Elsewhere)	Ref	10	MZY100021	Nut	1
2	MZY100139	Heater (M7-001, M7-004, M7-005)	1	11	MZY100149	Spacer	1
		[Includes Items 3 and 4]	1		MZY100151	Rocker Assembly (Includes Item 6, Item 15, Item 14)	1
	MZY100161	Heater (M7-002) [Includes Items 3]	1	12	MZY100152	Screw	1
	MZY100162	Heater (M7-003) [Includes Items 3]	1	13	MZY100163	Rocker Bracket	1
3	MZY100142	Nut	2	14	MZY100153	Locknut	2
4	MZY100143	Gasket	2	15	MZY100041	Screw	1
5	MZY100144	Thermostat	1	16	MZY100155	Bracket	1
6	MZY100145	Bracket	1	17	MZY100164	Reset Button	1
7	MZY100146	Washer	1	18	MZY100165	Rod	1
8	MZY100147	Lockwasher	2	19		Front Panel (Refer to "Front Panel Components" Elsewhere)	Ref
9	MZY100148	Spacer	1				

Always Specify Model & Serial Number