


Tank Assembly

SECTION VI PARTS LIST

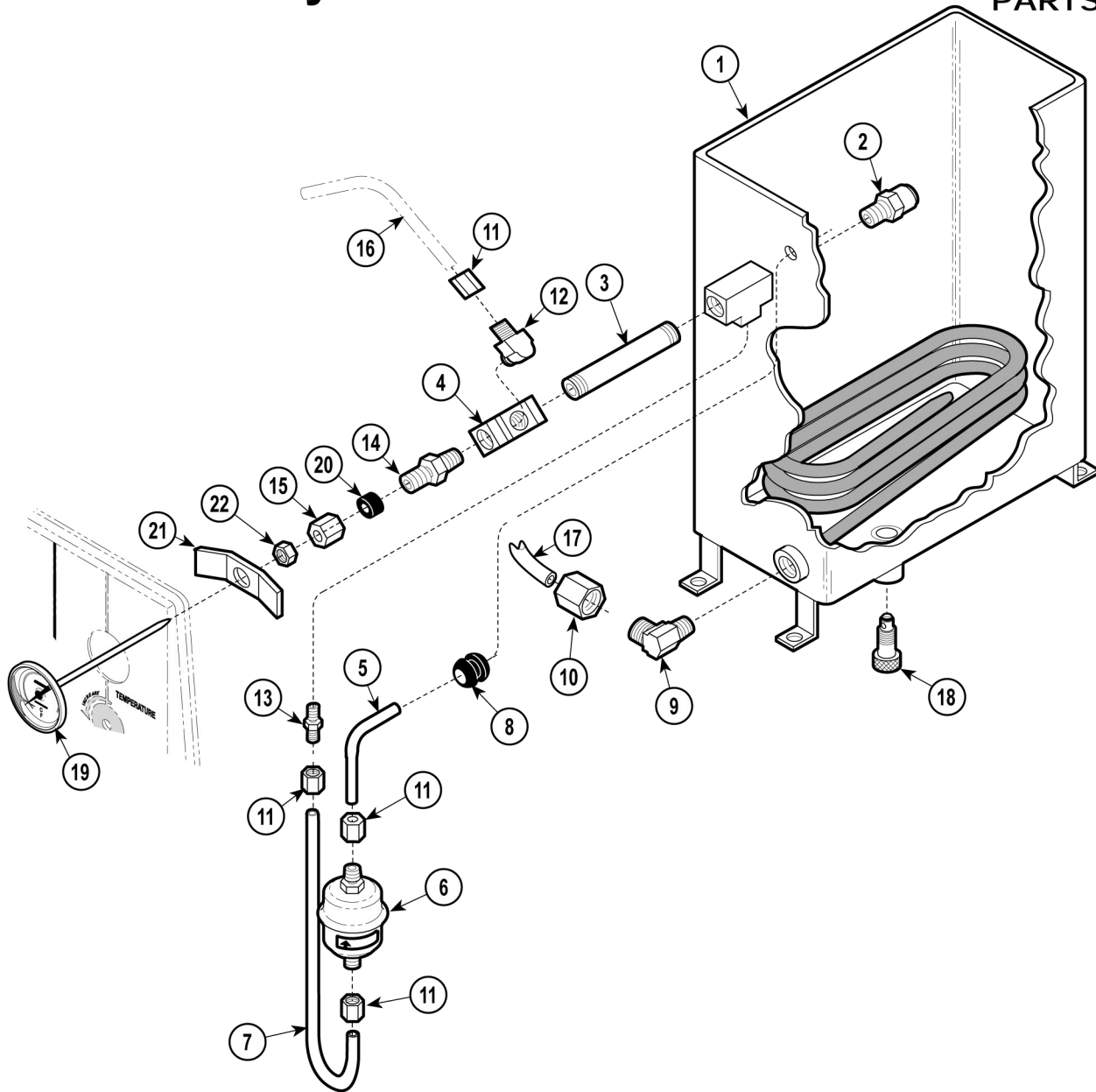

MA240805

Used on Units with Serial Numbers:
CR1114 thru CR1157 CS1570 thru CS1975
RB7194 thru RB7986

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	MZY100185	Tank Assembly(Incls. Items 1 thru 21) ..	1	12	MZY100175	Male Elbow	1
1	MZY100186	Tank	1	13	MZY100135	Male Connector	1
2	MZY100170	Safety Valve (Apply Teflon Tape #014-0026-00)	1	14	MZY100179	Connector Drilled	1
3	MZY100171	Pipe Nipple	1	15	•	• Compression Nut	1
4	MZY100172	Pipe "T"	1	16	MZY100039	Tube	Ref
5	MZY100187	Tube	1	17		Tube (Refer to "Sterilizer Main Components" Item 23)	6-5.1
6	MZY100174	Bellows	1	18	MZY100182	Drain (Includes O-Ring)	1
7	MZY100188	Tube	1		MZY100183	O-ring	1
8	MZY100189	Grommet	1	19	MZY100052	Thermometer (Includes Item 22)	1
9	MZY100137	Male Elbow	1	20	MZY100184	Thermometer Sleeve	1
10	MZY100178	Compression Nut	1	21	MZY100180	Bracket	1
11	MZY100136	Compression Nut	4	22	MZY100184	Thermometer Nut	1

Always Specify Model & Serial Number