


Tank Assembly

SECTION VI PARTS LIST

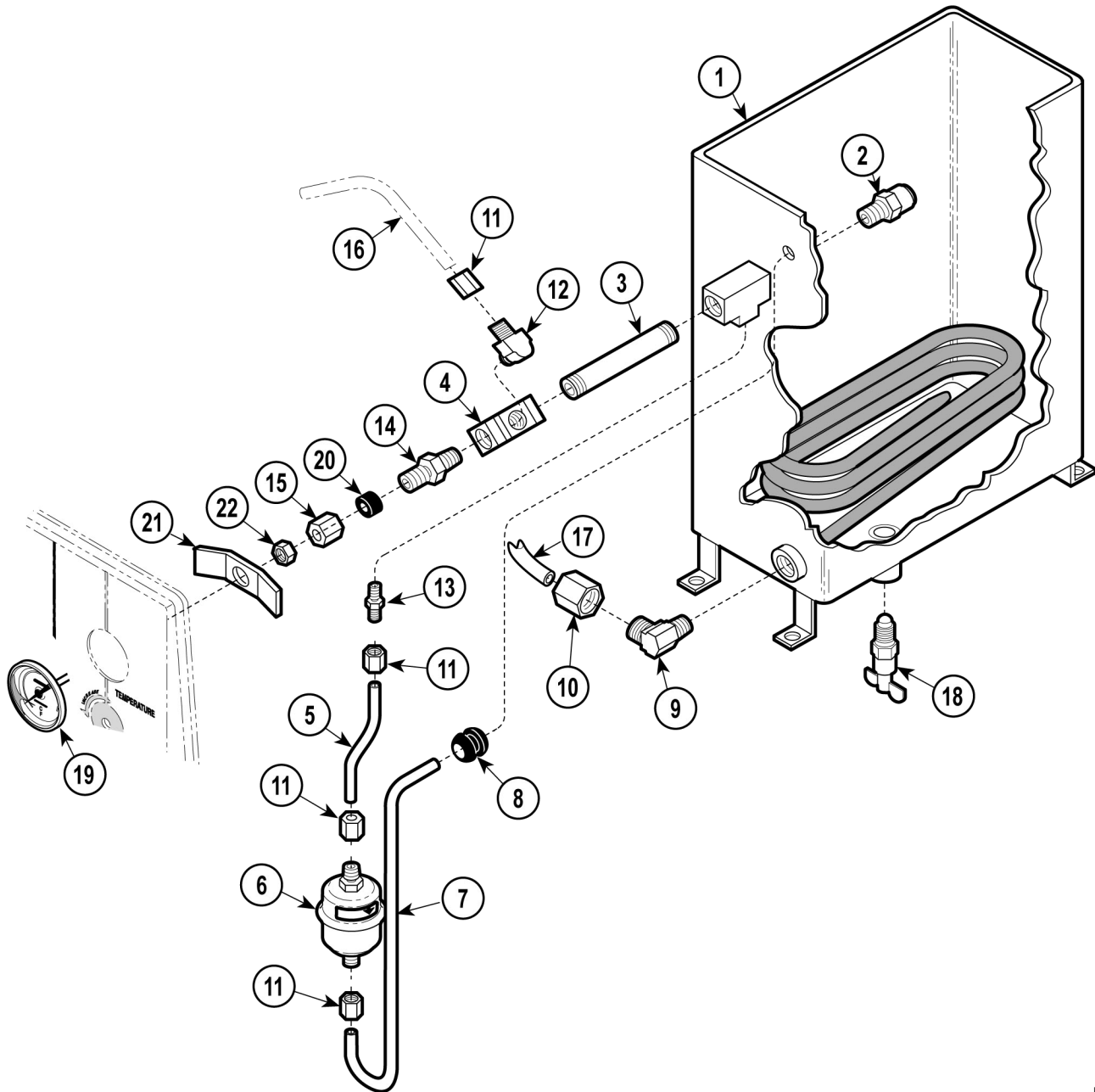

MA240807

Used on Units with Serial Numbers RB17993, CR1686, DR1268, CP1150, CS7754, FM1426 and FN1230 thru Present

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	MZY100192	Tank Assembly (Incls. Items 1 thru 21) .	1	12	MZY100175	• Male Elbow	1
1	MZY100193	• Tank	1	13	MZY100135	• Male Connector	1
2	MZY100170	• Safety Valve (Apply Teflon Tape #014-0026-00)	1	14	MZY100179	• Connector Drilled	1
3	MZY100171	• Pipe Nipple	1	15	• •	• • Compression Nut	1
4	MZY100172	• Pipe "T"	1	16		Tube (Refer to "Sterilizer Main Components" Elsewhere)	Ref
5	MZY100190	• Tube	1	17		Tube (Refer to "Sterilizer Main Components" Item 23)	6-5.3
6	MZY100174	• Bellows	1	18	MZY100194	Drain-Cock Replacement Kit	1
7	MZY100191	• Tube	1	19	MZY100052	Thermometer (Includes Item 26)	1
8	MZY100189	• Grommet	1	20	MZY100184	• Thermometer Sleeve	1
9	MZY100137	• Male Elbow	1	21	MZY100180	Bracket	1
10	MZY100178	• Compression Nut	1	22	MZY100181	Thermometer Nut	1
11	MZY100136	• Compression Nut	4				

Always Specify Model & Serial Number