


Valve Assembly

SECTION VI PARTS LIST

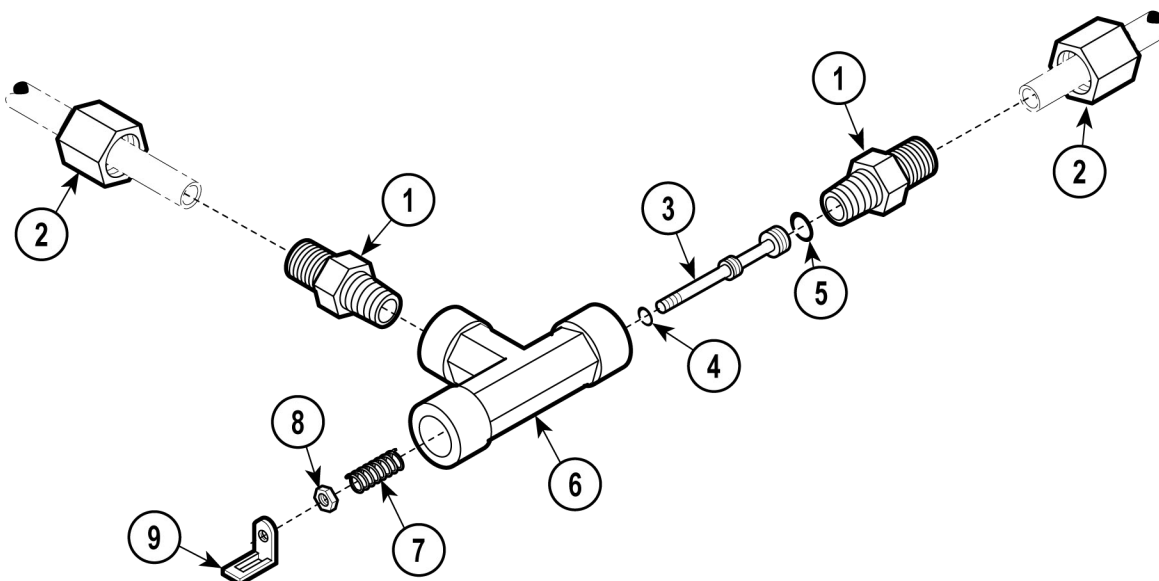

MA240603

Used On Units Built After 6-21-90

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	MZY100203	Valve Assembly	1			High Temp Lube #064-0002-01)	1
1	• MZY100196	• Fitting	2	6	• MZY100199	• Valve Body	1
2	• MZY100178	• Compression Nut	1	7	• MZY100200	• Valve Spring	1
3	• MZY100197	• Plunger	1	8	• MZY100201	• Nut	1
4	• MZY100183	• O-Ring (Apply High Temp Lube #064-0002-01)	1	9	• MZY100202	• Closing Bracket	1
5	• MZY100198	• O-Ring (Apply					

Always Specify Model & Serial Number